www.drivemedical.com

driveTM

bariatric extra large drop arm commode

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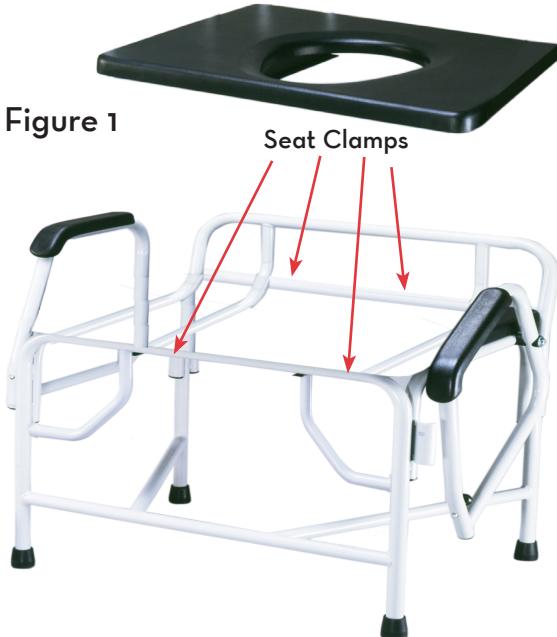
EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

driveTM

commode assembly instructions and precautions

WEIGHT CAPACITY: 1000 LBS.

assembling commode seat & lid:

1. Before assembling, allow toilet seat and lid to reach room temperature if exposed to cold. This will help prevent the seat and lid clamps from cracking while being assembled onto commode back frame rail.
2. Align the toilet seat clamps onto the back frame rail. (Both clamps should sit outside the pail brackets.) Push down on seat until clamps snap firmly into place.
3. Slide pail in from front of commode.

additional precautions

1. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a healthcare, dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
4. Ensure ALL screws, nuts and/or bolts are tightened.
5. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
6. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
7. Users with limited physical strength should be supervised or assisted while using commode.

EXAMINE THE FOLLOWING ANNUALLY

- Make sure the leg support buttons are securely through their height adjustment holes
- Examine the unit for loose, missing, or damaged nuts, bolts, screws, knobs. Tighten or replace as necessary.

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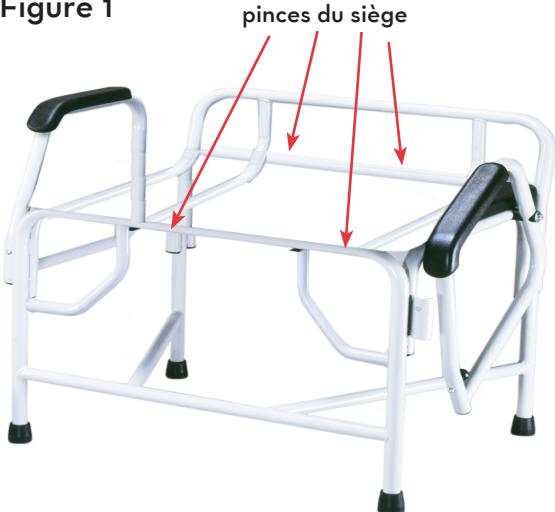
lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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The logo consists of the word "drive" in a lowercase, bold, sans-serif font. The letter "d" is blue, the "r" is orange, the "i" is blue, the "v" is blue, and the "e" is blue. A small trademark symbol (TM) is located at the top right of the "e".

© 2012 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

The logo consists of the word "drive" in a lowercase, bold, sans-serif font. The letter "d" is blue, the "r" is orange, the "i" is blue, the "v" is blue, and the "e" is blue. A small trademark symbol (TM) is located at the top right of the "e".

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The logo consists of the word "drive" in a bold, lowercase, sans-serif font. The letter "d" is blue, the "r" is orange, the "i" is white, the "v" is blue, and the "e" is white. A small "TM" symbol is located at the top right of the "e".

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© 2012 Medical Depot, Inc. Todos los derechos reservados.
Drive es una marca registrada de Medical Depot, Inc.
© 2012 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.