Drive Medical Design and Manufacturing
99 Seaview Boulevard,
Port Washington, NY 11050
Contact Us: (877) 224-0946

REV1.10.2.14

User manual

English

Contents

2	General Description	8	Tips for Skin Care
2	Intended Use	8	Contraindications
3	Included in this Package	9	Warnings
4	Instructions for Use	10	Precautions
6	Maintenance, Transportation and Storage	12	Adverse Reactions
6	Suggested for Pain Relief	12	Troubleshooting
7	Application Duration	14	Important Information Regarding Electromagnetic Compatibility (EMC)
7	Electrode Placement		

GENERAL DESCRIPTION

The **drive** Wireless TENS/EMS System is a battery operated wireless pulse generator that sends electrical impulses from a REMOTE control to RECEIVER (round) with ELECTRODE attached to them that are placed on the body to stimulate the nerves causing pain. When this happens, the nerves “FEEL” gentle electrical sensations instead of the pain.

The Wireless REMOTE controls the ON/OFF of the system, the INTENSITY and the MODE functions of the RECEIVER. The RECEIVER with the electrode attached to it adheres easily and firmly to the skin.

Because the System is Wireless - Needing No Lead Wires - you must use only DRIVE SNAP PIN Electrodes. Lead wire electrodes will not work with this Wireless system.

INTENDED USE


To be used for the temporary relief of pain associated with sore or aching muscles of the lower back, arms, or legs due to strain from exercise or normal household and work activities.

To be used for the symptomatic relief and management of chronic, intractable pain and relief of pain associated with arthritis.

INCLUDED IN THIS PACKAGE

- 1 REMOTE Control
- 2 RECEIVERS - Numbered 1 and 2
- LARGE LONG-LASTING BACK ELECTRODE, 1/ea.
- 2"x3" LONG-LASTING ELECTRODES, 2/ea.
- AC Adapter
- USB Cable for Recharging
- Instruction Booklet

- | | |
|-----------------------|-----------------------------------|
| 1. ON/OFF | 9. Working Indicator Light(Green) |
| 2. USB Port | /Charging Indicator Light(Red) |
| 3. LCD Screen | 10. Reset Button |
| 4. Intensity Adjustor | 11. USB Port |
| 5. Channel Switch | 12. AC Adapter |
| 6. Mode Selector | 13. Electrode Pad |
| 7. Power Switch | 14. USB Cable |
| 8. Receiver | |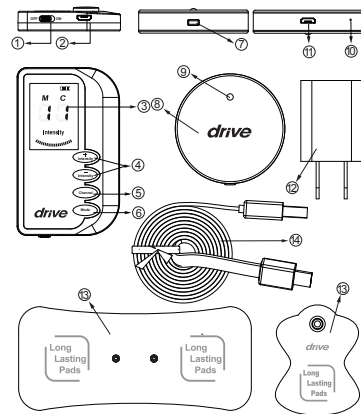


INSTRUCTIONS FOR USE

The REMOTE control is the sleek looking handset. It has a LCD display, Intensity Up + and Down - and C button for choosing RECEIVER/ Channel 1 or RECEIVER /Channel 2. The M button is to control the MODE functions of the 2 RECEIVERS - Channel 1 and Channel 2. *The RECEIVERS or Channels 1 and 2 are clearly marked.

ON/OFF

The slide SWITCH on the BOTTOM of the REMOTE is to turn it "ON" and "OFF". Its LCD display lights up when the REMOTE is "ON". If the LCD display is blank, the unit is "OFF" or the REMOTE and/or the RECEIVER(S) need to be recharged.

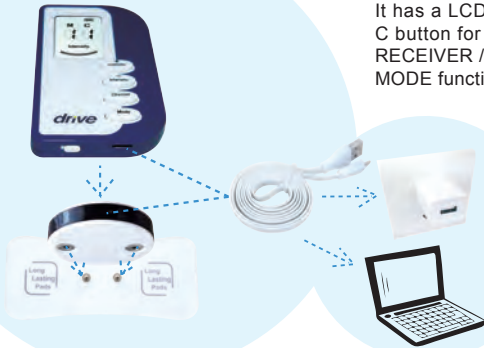INTENSITY

INTENSITY adjusting on the REMOTE is just below the LCD Screen figured as + and -. The more you push + the HIGHER "UP" the Intensity level of the Channel shown in the LCD display goes. The more you push - the LOWER "DOWN" the Intensity level of the Channel shown in the LCD display goes.

ROUND RECEIVERS

There are 2 RECEIVERS labeled as 1 and 2. *RECEIVER 1 is Channel 1 and RECEIVER 2 is Channel 2.

Both Channels are controlled by the REMOTE. To use Channel 1 push the "C" button on the REMOTE until you see 1 in the LCD display. Then set the MODE for it by pushing "M" on the REMOTE to see 1, 2, 3, 4 in the LCD display. To use Channel 2 push the "C" button on the REMOTE until you see 2 in the LCD display. Then set the MODE for it by pushing "M" on the REMOTE to see 1, 2, 3, 4 in the LCD display.

You can use both RECEIVERS for **Dual Channel** or only RECEIVER 1 for **Single Channel**.

CHANNEL or RECEIVER CONTROLS

When you push the power switch on each RECEIVER, you can see the light as green, and when you push it again the green light goes off.

MODES - PRE-PROGRAMMED MODE FUNCTIONS

Make sure you first set the unit to Channel 1 or Channel 2 by pushing “C” and seeing 1 or 2 appear in the LCD display. Then push the “M” button on the REMOTE to change the selected Channel from Mode1, to Mode 2, to Mode 3 or to Mode 4. The MODE number being used in the selected Channel will appear in the LCD screen as 1, 2, 3, 4. **(See the description of SUGGESTED MODES for different Pain areas and types in this booklet).**

MAINTENANCE, TRANSPORTATION AND STORAGE

1. Non-flammable cleaning solution is suitable for cleaning the device.
2. Stains and spots can be removed with a cleaning agent.
3. Do not submerge the device in liquids or expose it to large amounts of water.
4. Return the device to the carrying box to ensure that the unit is well-protected before transportation.

SUGGESTED FOR PAIN RELIEF

1. Legs, Arms, Hands, Feet - Mode 1, Mode 3
2. Knees - Mode1, Mode 3

3. Low Back, Hip, Stomach - Mode 1, Mode 2, Mode 3
4. Shoulders, Upper Back - Mode 1, Mode 2, Mode 3
5. Neck - Mode1, Mode 2
6. Combination Wave Mode 4 is a mixture of Mode 1, Mode 2 and Mode 3 and works well on all Pain Conditions

APPLICATION DURATION

2-3 times/day, 45 minutes/time, over 10 days for one course.

ELECTRODE PLACEMENT

The placement of electrodes can be one of the most important parameters in achieving success with TENS therapy. Of utmost importance is the willingness of the user to try the various styles of electrode placement to find which method best fits the his or her needs.

Every user responds to electrical stimulation differently and their needs may vary from the conventional settings suggested here. If the initial results are not positive, feel free to experiment.

TIPS FOR SKIN CARE

To avoid skin irritation, especially if you have sensitive skin, follow these suggestions:

1. Wash the area of skin where you will be placing the electrodes, using mild soap and water before applying electrodes, and after taking them off. Be sure to rinse soap off thoroughly and dry skin well.
2. Excess hair may be clipped with scissors; do not shave stimulation area.
3. Wipe the area with the skin preparation your clinician has recommended. Let this dry. Apply electrodes as directed.
4. Many skin problems arise from the “pulling stress” from adhesive patches that are excessively stretched across the skin during application. To prevent this, apply electrodes from centre outward; avoid stretching over the skin.
5. When removing electrodes, always remove by pulling in the direction of hair growth.
6. It may be helpful to rub skin lotion on electrode placement area when not wearing electrodes.
7. Never apply electrodes over irritated or broken skin.

CONTRAINDICATIONS

DO NOT use this device if you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference, or death.

WARNINGS

1. If you are in the care of a physician, consult with your physician before using this device;
2. If you have had medical or physical treatment for your pain, consult with your physician before using this device;
3. If your pain does not improve, becomes more than mild, or continues for more than five days, stop using the device and consult with your physician;
4. Do not apply stimulation over your throat because this could cause severe muscle spasms resulting in closure of your airway, difficulty in breathing, or adverse effects on heart rhythm or blood pressure;
5. Do not apply stimulation across your chest because the introduction of electrical current into the chest may cause rhythm disturbances to your heart, which could be lethal;
6. Do not apply stimulation over open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions (e.g., phlebitis, thrombophlebitis, varicose veins);
7. Do not apply stimulation over, or in proximity to, cancerous lesions;
8. Do not apply stimulation in the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when the electrical stimulation device is in use;
9. Do not apply stimulation when in the bath or shower;
10. Do not apply stimulation while sleeping;
11. Do not apply stimulation while driving, operating machinery, or during any activity in which

- electrical stimulation can put you at risk of injury; and
12. Do not use the device on children, if it has not been evaluated for pediatric use.
 13. Consult with your physician before using this device, because the device may cause lethal rhythm disturbances to the heart in susceptible individuals; and
 14. Apply stimulation only to normal, intact, clean, healthy skin.

PRECAUTIONS

1. TENS is not effective for pain of central origin, including headache;
2. TENS is not a substitute for pain medications and other pain management therapies;
3. TENS devices have no curative value;
4. TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism;
5. Effectiveness is highly dependent upon patient selection by a practitioner qualified in the management of pain patients.
6. The long-term effects of electrical stimulation are unknown;
7. Since the effects of stimulation of the brain are unknown, stimulation should not be applied across your head, and electrodes should not be placed on opposite sides of your head.

8. The safety of electrical stimulation during pregnancy has not been established, consult with your physician;
9. You may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium (gel);
10. If you have suspected or diagnosed heart disease, you should follow precautions recommended by your physician; and.
11. If you have suspected or diagnosed epilepsy, you should follow precautions recommended by your physician.
12. Use caution if you have a tendency to bleed internally, such as following an injury or fracture;
13. Consult with your physician prior to using the device after a recent surgical procedure, because stimulation may disrupt the healing process;
14. Use caution if stimulation is applied over the menstruating or pregnant uterus; and
15. Use caution if stimulation is applied over areas of skin that lack normal sensation. The user manual should advise users of the following:
 - Keep this device out of the reach of children; and
 - Use this device only with the leads, electrodes and accessories recommended by the manufacturer.
16. Do not use TENS anywhere on the head or face (unless instructed by your therapist for TMJ), front of the neck, internally, or on genitals.

ADVERSE REACTIONS

You may experience skin irritation and burns beneath the stimulation electrodes applied to your skin; You may experience headache and other painful sensations during or following the application of electrical stimulation near your eyes and to your head and face; and You should stop using the device and should consult with your physician if you experience adverse reactions from the device.

TROUBLESHOOTINGS

1. The REMOTE is not turning on or I can't see anything on the screen.
 - Make sure the REMOTE has been fully charged.
2. The power light of RECEIVER flashes continuously.
 - Charge the RECEIVER soon or it will turn off automatically.
3. The REMOTE and/or RECEIVER do(es) not seem to be charging.
 - Ensure the USB cable is firmly in the REMOTE/RECEIVER on one side and plugged into the AC adapter into the electrical socket on the other side - Isolate whether there is a problem with the AC adapter by plugging the USB cable from the REMOTE/RECEIVER into a computer.
4. Adhesive electrode pads do not stick to skin even after cleaning and moistening their surface.
 - Replace the adhesive gel pads. Pads can be purchased from your distributor or from www.drivemedical.com.

5. During use the skin feels a painful burning sensation or the stimulation becomes weakened.
 - Apply a few drops of water to adhesive surface of the pads and make sure the pads are pressed firmly to the skin during application.
6. When pressing "M" (Mode) and "+" "-" (Intensity adjustor) of the REMOTE, there is no any reaction.
 - Make sure both REMOTE and RECEIVERS are turned on, and the Channel number matches the RECEIVER number correctly.
 - If still no any reaction, the user should match the REMOTE and RECEIVERS as to the following steps: Press the "C" button on the REMOTE for more than 3 seconds, you will see "C" and Number (1, 2 or 3) flashing, and then quickly double click the power switch of the RECEIVER. At this moment, you could see the Number on the REMOTE stops flashing, which means this RECEIVER is matched successfully. And also you could match the other RECEIVERS as to the above steps.
7. The REMOTE can't control the RECEIVER well.
 - Make sure the distance between the REMOTE and RECEIVER is less than 10 meters under the open environment.

IMPORTANT INFORMATION REGARDING ELECTROMAGNETIC COMPATIBILITY (EMC)

Drive PainAway Model# RTLAGF-900 Tens Conforms to EMC Standards

As the number of electronic devices such as computers and cellular mobile telephones increases, medical devices in use may be at risk to electromagnetic interference from other devices. Electromagnetic interference may result in faulty operation of the medical device and create a potentially unsafe situation. Medical devices should also not interfere with other devices. In order to regulate the requirements for EMC or Electromagnetic Compatibility with the goal to prevent unsafe product use, the EC60601-1-2 standard was implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices.

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