

three year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for three years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Three Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite*Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

seat

item # KA 1285

for use with nimbo, tyke
(item # KA 1200)



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

*Wenzel*ite*Re/hab*

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation

1. Remove bolts (A).
2. Drop outside frame connecting tube (B)
3. Lift horizontal U-tube (E).
4. Slide seat mounting brackets (C) onto horizontal U-tube and push towards the rear.
5. Tighten knobs (D) under seat mounting brackets.
6. Reinstall bolts (A).

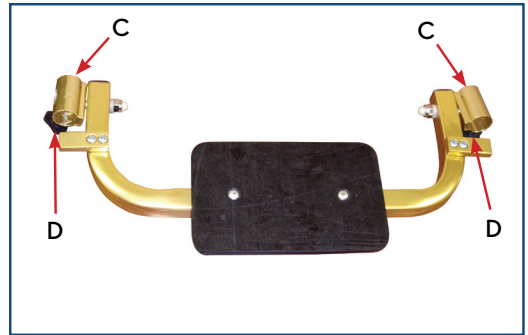


figure 2

operation

Flip up seat when standing and walking and fold down seat for sitting and resting. Seat should be in the up position when folding walker.



figure 3

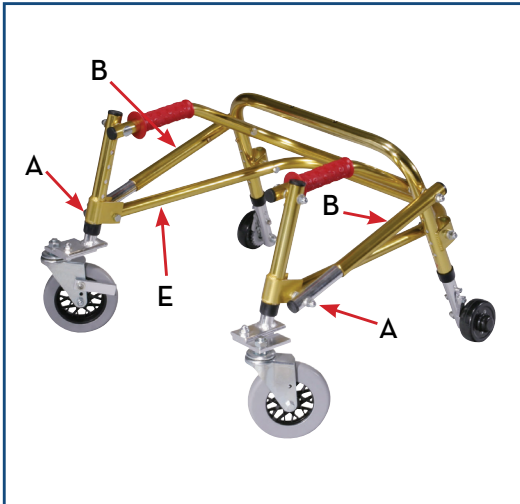


figure 1

three year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for three years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Three Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite*Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

seat

item # KA 2285

for use with nimbo, junior
(item # KA 2200)



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

*Wenzel*ite*Re/hab*

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation

1. Remove and discard bolts (A) on rear U-Bar.
2. Remove all mounting hardware (C) from sliding bracket.
3. Insert mounting tubes (B) into opening on the bottom of rear U-Bar (D) and align U-Bar and mounting tube holes.
4. Install mounting hardware (C) at position where bolts (A) were removed. When assembled, the hardware components should be positioned in the following order: Starting from the inside of the walker - bolt, washer, sliding bracket, washer, nut, black plastic washer, U-Bar, black plastic washer, outside frame connecting tube, black washer and end nut (figure 3).
5. Tighten nuts and allow for slight play for the sliding bracket.

operation

Flip up seat when standing and walking and fold down seat for sitting and resting. Seat should be in the up position when folding walker.

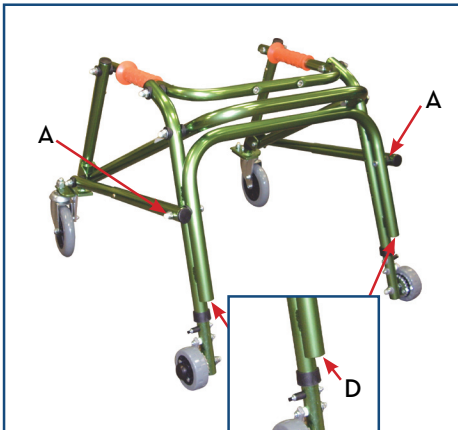


figure 1

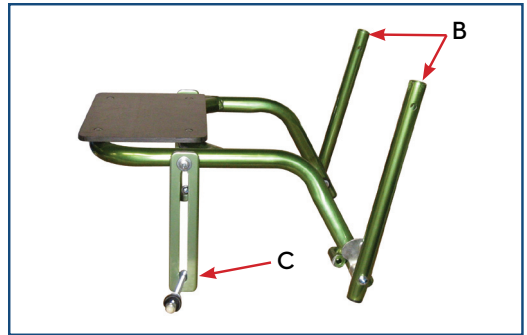


figure 2

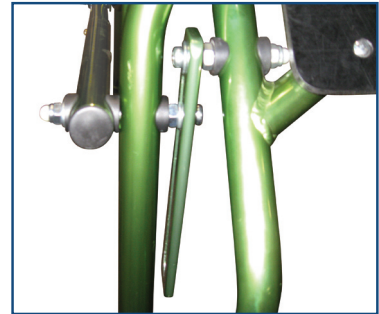


figure 3



figure 4

three year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for three years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Three Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite*Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

seat

item # KA 3285

for use with nimbo, youth
(item # KA 3200)



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

*Wenzel*ite*Re/hab*

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation

1. Remove bolts (A) on rear U-Bar.
2. Remove all mounting hardware (C) from sliding bracket.
3. Insert mounting tubes (B) into opening on the bottom of rear U-Bar (E), align U-Bar and mounting tube holes, insert bolts (A) and tighten.
4. Remove and discard bolts (D). Replace with mounting hardware (C). When assembled, the hardware components should be positioned in the following order: Starting from the inside of the walker - bolt, washer, sliding bracket, washer, nut, black plastic washer, U-Bar, black plastic washer, outside frame connecting tube, black washer and end nut (figure 3).
5. Tighten nuts and allow for slight play for the sliding bracket.

operation

Flip up seat when standing and walking and fold down seat for sitting and resting. Seat should be in the up position when folding walker.

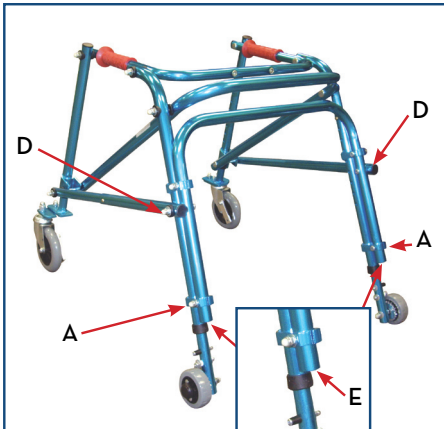


figure 1

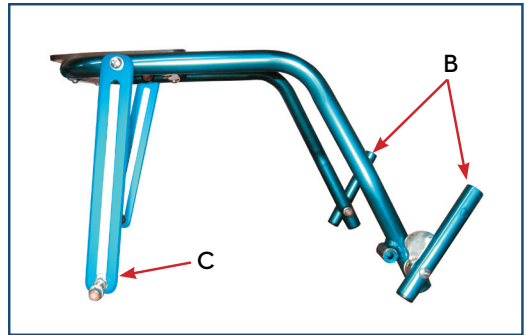


figure 2



figure 3



figure 4

three year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for three years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Three Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite*Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

seat

item # KA 4285

for use with nimbo, young adult
(item # KA 4200)



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

*Wenzel*ite*Re/hab*

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation

1. Remove bolts (A) on rear U-Bar.
2. Remove all mounting hardware (C) from sliding bracket.
3. Insert mounting tubes (B) into opening on the bottom of rear U-Bar (E), align U-Bar and mounting tube holes, insert bolts (A) and tighten.
4. Remove and discard bolts (D). Replace with mounting hardware (C). When assembled, the hardware components should be positioned in the following order: Starting from the inside of the walker - bolt, washer, sliding bracket, washer, nut, U-Bar, washer and end nut (figure 3).
5. Tighten nuts and allow for slight play for the sliding bracket.

operation

Flip up seat when standing and walking and fold down seat for sitting and resting. Seat should be in the up position when folding walker.

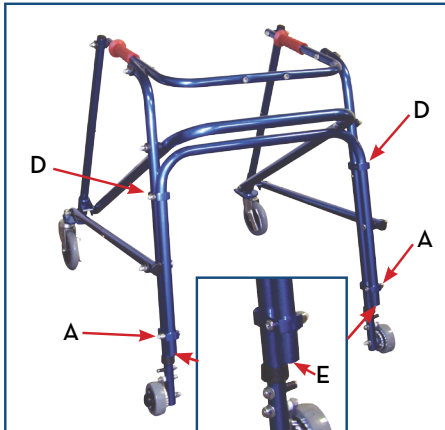


figure 1

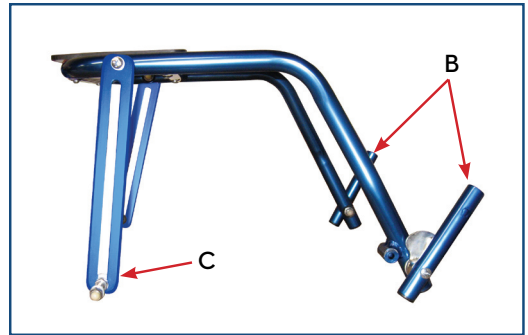


figure 2



figure 3



figure 4

three year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for three years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Three Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



Wenzel^{ite}Re/hab

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

seat

item # KA 5285

for use with nimbo, adult
(item # KA 5200N)



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

Wenzel^{ite}Re/hab

www.drivemedical.com

drive
WWW.DRIVEMEDICAL.COM

installation

1. Remove mounting hardware (A).
2. Mount seat tube into receiving hole (D) on rear U-Bar and install hardware as per Diagram 1.
3. Remove mounting hardware (B).
4. Mount seat tube into receiving hole (E) on rear U-Bar and install hardware as per Diagram 2.
PLEASE NOTE: Aluminum plate (G) must be positioned facing upwards. This is to ensure that the seat stays in the upright position when walking.
5. Remove nut and black plastic washer (C) on one side and install into receiving hole (F) on rear U-Bar as per Diagram 3.
6. Tighten nut but allow sufficient play for the sliding bracket.
7. Repeat on other side.

operation

Flip up seat when standing and walking and fold down set for sitting and resting. Seat should be in the up position when folding walker.

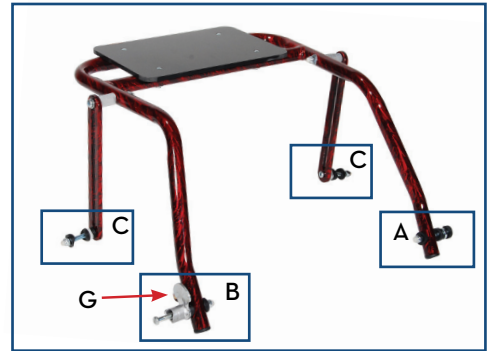
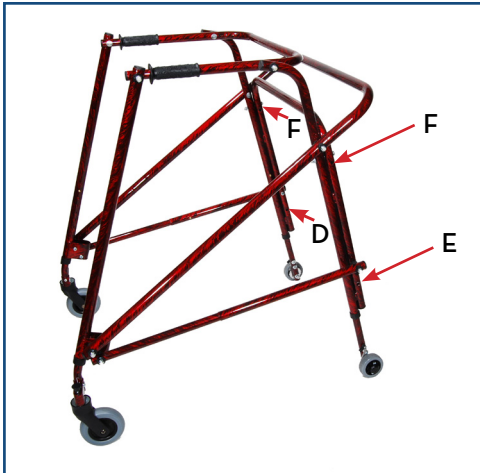


diagram 1



diagram 2



diagram 3