## DEALER/PROFESSIONAL INFORMATION

### PRODUCT DESCRIPTION

The OXYMIZER oxygen-conserving devices are disposable products used in the routine nasal administration of oxygen. The OXYMIZER Pendant with the reservoir in a pendant offers these unique features:

- A close-coupled reservoir that fills with 18ml of expired air at the initial phase of expiration, which is enriched or replaced with oxygen during the remainder of expiration.
- A diaphragm, which immediately collapses on initial inspiration, delivering a bolus of highly oxygen-enriched air to the deepest portion of the lungs.
- Soft nasal prongs, which offer maximum comfort and place less pressure on the septum.

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| Oxygen Requirements<br>With Standard<br>Nasal Cannula | Oxygen Requirements<br>With P-224 OXYMIZER<br>Devices | Resulting Oxygen<br>Savings |
|---|---|-----------------------------|
| 2 lpm   | 0.5 lpm   | 75.00%                      |
| 3 lpm   | 1.0 lpm   | 66.60%                      |
| 3.5 lpm   | 1.5 lpm   | 57.14%                      |
| 4 lpm   | 2.0 lpm   | 50.00%                      |
| 5 lpm   | 2.5 lpm   | 50.00%                      |
| 5.5 lpm   | 3.0 lpm   | 45.45%                      |
| 6.0 lpm   | 3.5 lpm   | 41.67%                      |
| 6.5 lpm   | 4.0 lpm   | 38.46%                      |
| 7.0 lpm   | 4.5 lpm   | 35.71%                      |
| 7.5 lpm   | 5.0 lpm   | 33.33%                      |

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CODE #P-224: Box of 24 individually packaged OXYMIZER  
Pendant oxygen-conserving devices

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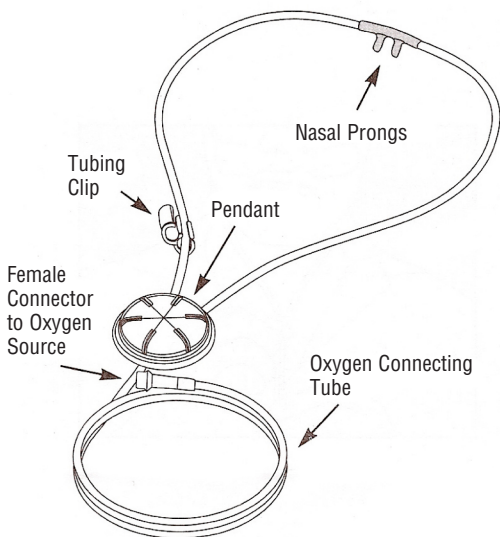
CE

© 2013 CHAD Therapeutics  
U.S. Pat Pending  
PM-00051/13/B

## USER DIRECTIONS:

Your doctor or healthcare professional has prescribed the OXYMIZER oxygen-conserving device for you. Used properly, this device will provide you with the oxygen you need, as well as these added benefits:

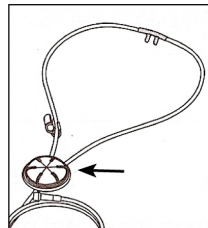
- It will take a lower flow rate to get the same amount of oxygen into your body, which can be as much as 75% less (depending on the flow rate) than a standard nasal cannula.
- If you use a portable oxygen system, it will last longer and allow you to be away from your main oxygen source for a longer period of time; or, you may be able to use a smaller, lighter weight portable system.

It is important that you follow your physician's instructions carefully concerning:

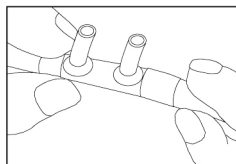
- How often and how long you receive oxygen from the system.
- The flow rate to use.  
*(Note: The flow rate indicated for use with the OXYMIZER device may be 1/4 - 1/2 of the amount prescribed with a non-oxygen-conserving cannula.)*
- When to replace your OXYMIZER device.

## HOW TO PUT ON THE OXYMIZER PENDANT

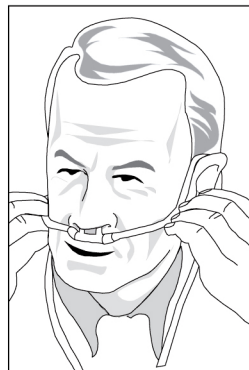
**1.** Uncoil the tubing completely. Hold the device so that the front of the Pendant is facing away from you.  
*(Note: The front of the Pendant is designated with slots.)*

**2.** Position the nasal prongs so that they fit comfortably into your nostrils. The prongs should extend well into and point toward the back of your nose.

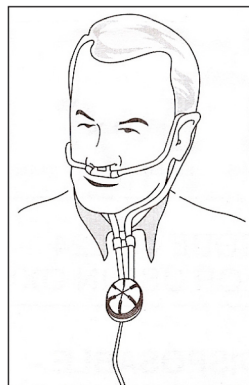
**3.** Position the nasal prongs and extend the tubing across your cheekbones.

**4.** Place the tubing around your ears and adjust each side so that you have a snug, comfortable fit.

**5.** Attach the tubing clip to join both sides of the cannula tubing. Slide the clip upward beneath your chin to hold the tubing in place (you may choose not to use the clip simply by removing it).

**6.** The Pendant portion should rest against your chest below your throat. You may want to tuck it under your shirt or blouse.

**7.** Connect the female connector to your oxygen source.

**8.** Turn on the oxygen and set the flow rate as prescribed by your doctor or health care professional. *(NOTE: The flow rate prescribed for use with this device may be 1/4-1/2 that prescribed with a non-oxygen-conserving cannula.)*

**REPLACEMENT:**

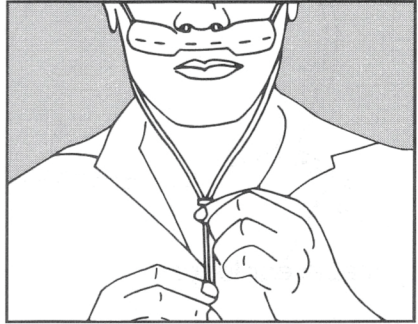
The OXYMIZER Pendant device should be replaced after approximately three weeks. More frequent replacement may be needed for cleanliness and sanitation, depending on conditions of use.

|               |             |
|---------------|-------------|
| TODAY'S DATE: | REPLACE ON: |
|               |             |

**HUMIDIFICATION UNNECESSARY**

Since less total oxygen is passing over the nasal mucosa, the drying effect of oxygen is significantly less and supplementary humidification can usually be eliminated. Since excessive moisture could impede the membrane action of the OXYMIZER Pendant device, use with humidifiers should be avoided.

You may also be interested in the original OXYMIZER oxygen-conserving device with the oxygen-saving mechanism contained in a facepiece, which is ideal for use at home (see illustration below).

# OXYMIZER® PENDANT

DISPOSABLE OXYGEN-CONSERVING DEVICES

CODE #P-224  
FOR USE IN OXYGEN THERAPY

**DISPOSABLE-  
FOR SINGLE PATIENT USE ONLY**  
DIRECTIONS FOR USE ENCLOSED

CONTENTS: One (1) OXYMIZER Pendant disposable oxygen-conserving device with curved tubular nasal prongs, six (6) feet of connecting tube, and a female connector.

Rx only

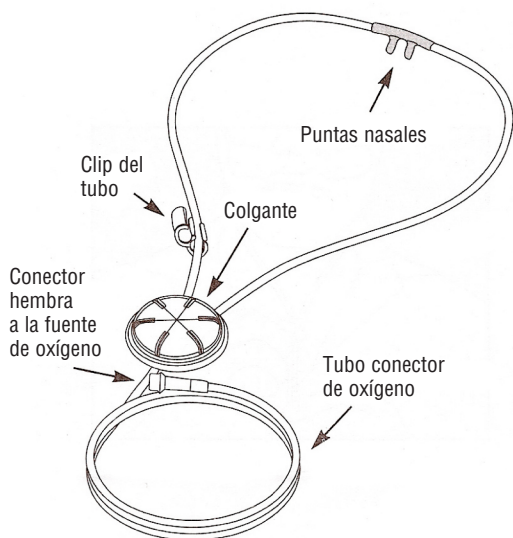
Lot#

Manufactured for:

Inovo, Inc  
401 Leonard Blvd. North  
Lehigh Acres, FL 33971  
Toll-free: 877-224-0946  
www.chadtherapeutics.com  
www.drivemedical.com

© 2008 CHAD Therapeutics  
U.S. Patent Pending  
Made in China

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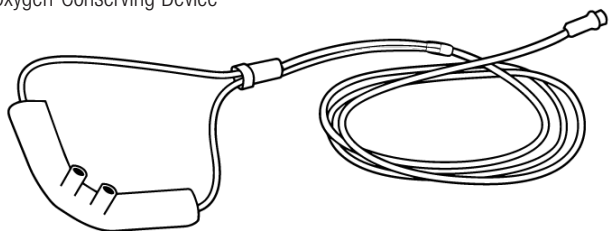
## DEALER/PROFESSIONAL INFORMATION

### PRODUCT DESCRIPTION

The OXYMIZER oxygen-conserving devices are disposable products used in the routine nasal administration of oxygen. The OXYMIZER with the oxygen-conserving reservoir in a facepiece offers these unique features:

- a) A close-coupled reservoir that fills with 18ml of expired air at the initial phase of expiration, which is enriched or replaced with oxygen during the remainder of expiration.
- b) A diaphragm, which immediately collapses on initial inspiration, delivering a bolus of highly oxygen-enriched air to the deepest portion of the lungs.
- c) Soft nasal prongs, which offer maximum comfort and place less pressure on the septum.

OXYMIZER Disposable  
Oxygen-Conserving Device

### F-224

As a result of their unique designs, the OXYMIZER devices efficiently administer oxygen while:

- Providing savings in oxygen usage of up to 75%, depending on flow rate
- Significantly reducing the cost of providing oxygen therapy
- Increasing the length of time that a patient utilizing a portable oxygen system can be away from his/her primary oxygen source
- Reducing nasal irritation
- Eliminating humidification, because lower oxygen flow rates reduce drying effect. Excessive moisture from humidifiers can also impede the membrane action of the device and should be avoided.

### INSTRUCTIONS FOR USE

Because the OXYMIZER devices are interactive with the patient, individual oxygen requirements with the OXYMIZER should be determined. Oximetry testing may be used to determine the best oxygen flow setting for each patient.

Usually, desirable blood oxygen saturation can be maintained with the OXYMIZER devices at much lower flow rates than required with a standard nasal cannula. Although the flow rates may vary from patient to patient, the following illustrates the savings seen in clinical evaluation:

| Oxygen Requirements<br>With Standard<br>Nasal Cannula | Oxygen Requirements<br>With F-224 Oxymizer<br>Device | Resulting Oxygen<br>Savings |
|---|--|-----------------------------|
| 2 lpm   | 0.5 lpm  | 75.00%                      |
| 4 lpm   | 1.5 lpm  | 62.50%                      |
| 6 lpm   | 2.5 lpm  | 58.33%                      |
| 8 lpm   | 4.5 lpm  | 43.75%                      |
| 10 lpm  | 6 lpm  | 40.00%                      |
| 12 lpm  | 10 lpm   | 16.67%                      |
| 15 lpm  | 12.5 lpm   | 16.67%                      |
| 20 lpm  | 16.5 lpm   | 17.5%                       |

Patient directions for use are provided  
with each OXYMIZER device.

### REPLACEMENT

The OXYMIZER and OXYMIZER Pendant devices should be replaced after approximately three weeks of use. More frequent replacement may be needed for cleanliness and sanitation, depending on conditions of use.

### CLEANING

The disposable OXYMIZER oxygen-conserving devices are clean and ready for use, and are supplied in individual containers. The outer surfaces may be cleaned with soap and water, but care must be taken not to allow the membrane within the reservoir to become wet.

### AVAILABILITY

The OXYMIZER devices are available to healthcare professionals, hospitals, and home care providers as follows:

CODE #F-224: Box of 24 individually packaged OXYMIZER  
oxygen-conserving devices

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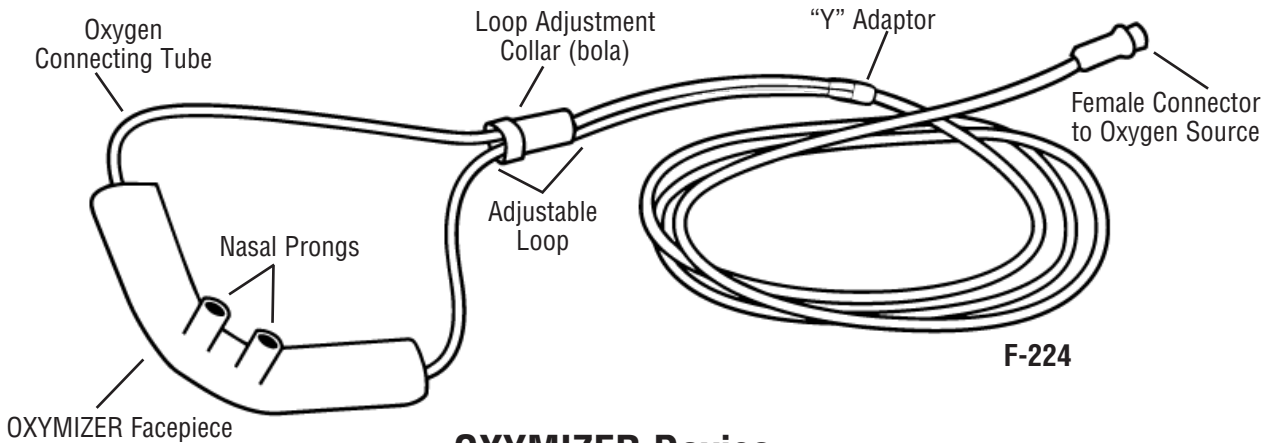
© 2013 CHAD Therapeutics  
U.S. Pat Pending  
PM-00058/12/A

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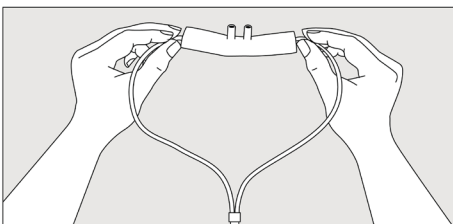
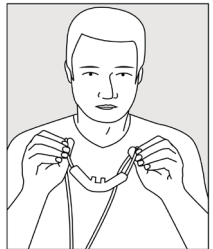
- How often and how long you receive oxygen from the system.
- The flow rate to use.  
(Note: The flow rate indicated for use with the OXYMIZER device may be 1/4 - 1/2 of the amount prescribed with a non-oxygen-conserving cannula.)
- When to replace your OXYMIZER device.

### HOW TO WEAR THE OXYMIZER DEVICE

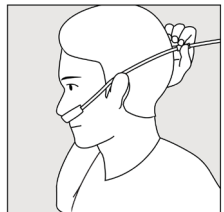
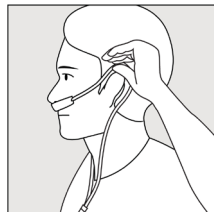
1. Slide the loop adjustment collar (bola) down to enlarge the size of the loop.

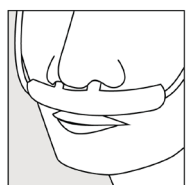
2. With the back of the facepiece toward you, pick up the OXYMIZER with one of the small flexible plastic tubes in each hand, as if putting on a pair of eyeglasses.

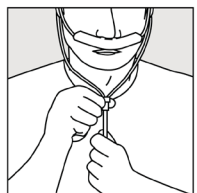
3. Place the flexible plastic tubing over your ears (as shown in diagram below) and under your chin or pass the tubing over your ears and around the back of your head.

4. The facepiece should rest on your upper lip under your nose with the oxygen-delivering prongs extending well into and pointing towards the back of your nose.

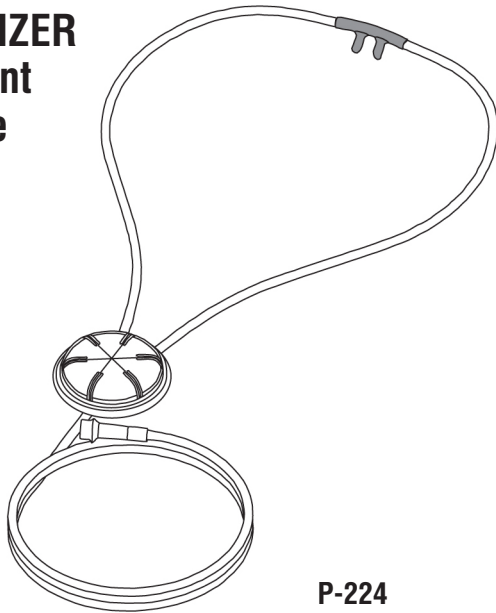
5. Slide the bola up toward your chin so that the cannula fits snugly and comfortably against your face.

6. Connect the female connector to your oxygen source.

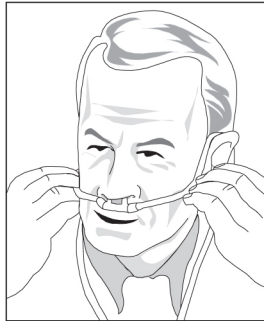
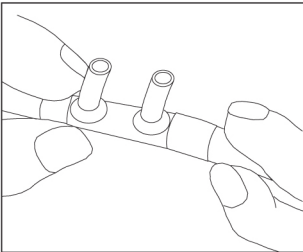
7. Turn on the oxygen and set the flow rate as prescribed by your doctor or healthcare professional. (Note: The flow rate prescribed for use with this device may be 1/4 - 1/2 of the amount prescribed with a non-oxygen-conserving cannula.)

**OXYMIZER  
Pendant  
Device**

**P-224**

You may also be interested in the OXYMIZER Pendant device (pictured above and below). In this version, the oxygen-conserving mechanism is contained within the pendant, which rests against your chest and can be easily concealed by your clothing.

**REPLACEMENT**

The OXYMIZER oxygen-conserving devices should be replaced after approximately three weeks. More frequent replacement may be needed for cleanliness and sanitation, depending on conditions of use.

|               |             |
|---------------|-------------|
| TODAY'S DATE: | REPLACE ON: |
|               |             |

**HUMIDIFICATION UNNECESSARY**

Since less total oxygen is passing over the nasal mucosa, the drying effect of oxygen is significantly less and supplementary humidification can usually be eliminated.

**CODE #F-224  
FOR USE IN OXYGEN THERAPY**

**DISPOSABLE-  
FOR SINGLE PATIENT USE ONLY  
DIRECTIONS FOR USE ENCLOSED**

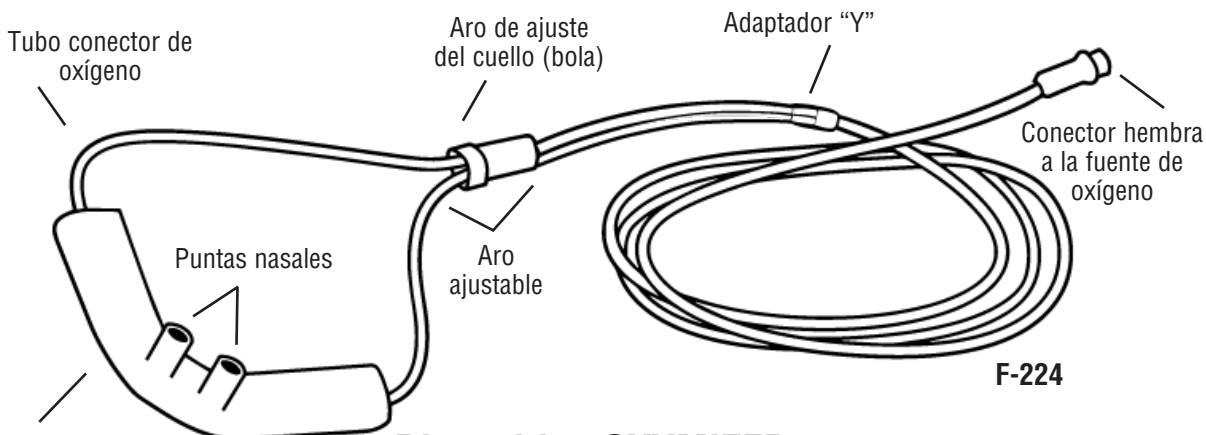
**CONTENTS:** One (1) OXYMIZER disposable oxygen-conserving device with curved tubular nasal prongs, 6 feet of connection tubing, and one (1) female connector.

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