

# one year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite Re/hab

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

trotter foot & ankle positioner

Item # TR 8028



EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP

*Wenzel*ite Re/hab

[www.drivemedical.com](http://www.drivemedical.com)

drive  
MEDICAL DESIGN & MANUFACTURING



## installation

1. Remove the four nuts and washers (Fig 1 A) from tips of mounting bracket.
2. Insert mounting brackets into four outer foot plate grooves (Fig 2)
3. Replace all washers and nuts on the underside of footplate and tighten.

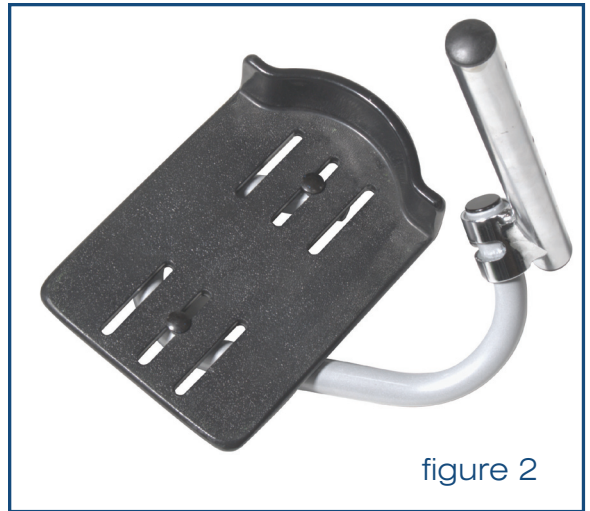


figure 2

## adjustments

1. Adjust length of front strap (Fig 1B) to fit user's instep.
2. Open ankle strap (Fig 1 C) to insert and position foot.
3. Adjust ankle strap to secure foot and close straps.

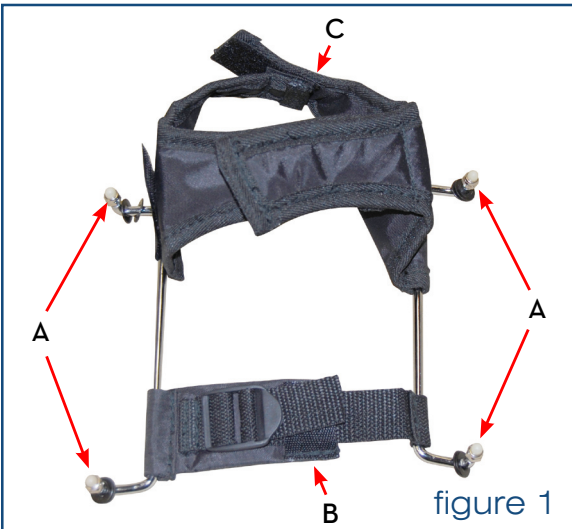


figure 1