

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2012 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

## IV Pole

for use with all Wenzelite  
safety rollers

Item # CE 1020

Wenzelite Re/hab<sup>TM</sup>  
drive

[www.drivemedical.com](http://www.drivemedical.com)

## installation instructions

1. Remove 2 brackets (A) from box.  
Remove bolts (B) from rectangular side of bracket. Position both brackets over Safety roller as in picture to the right.
2. Loosen bolts (C) and put bar (D) through round brackets. Tighten all bolts.
3. You can raise or lower top bar by loosening knob (E).

