

lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

width-adjustable
retractable seat

item # CE 1286

for use with CE 1000 XL,
CE 1000 B, CE 1000 BK,
CE 1200 BK and CE 1000 NBL



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

Wenzelite Re/habTM
drive

installation

Left T Bar

1. Mount bottom bracket (A) onto Safety Roller's left bottom frame and top bracket (B) onto upright. (Figure 2)
2. Insert screws into brackets (A & B).

Right T Bar with Seat

1. Mount bottom bracket onto Safety Roller's right bottom frame (C) and top bracket onto upright (D).
2. Insert screws (C & D) into brackets.

Note: Adjust handlebar height before tightening screws

operation

1. Unfold Safety Roller.
2. Lift handlebar and position over left upright.
3. Unhook seat from bottom frame and lift up.
4. Secure onto top of left T bar. (Figure 3)
5. To fold seat, lift off left T bar, lower and latch onto right bottom frame. (Figure 1)

figure 1



figure 2



figure 3