

12-1220/12-1221/12-1222

**BASELINE**<sup>®</sup> **EVALUATION  
INSTRUMENTS**

## **Finger Circumference Gauge**



**To measure circumference of a joint or digit:**

1. Wrap webbing around finger or joint.
2. Thread webbing through slot and align excess webbing across ruler.
3. Record measurement along ruler at end of webbing. This is the circumference.



Fabrication Enterprises Inc.  
P.O. Box 1500  
White Plains, NY 10602 (USA)  
tel: 1-914-345-9300  
fax: 1-914-345-9800  
[www.Fab-Ent.com](http://www.Fab-Ent.com)



Authorized CE representative:  
RMS UK Ltd.  
28 Trinity Road  
Nailsea, Somerset BS48 4NU  
(UK)

ver 6/11