

Advanced Wound Care Medical Tape Application Guide



Step 1: Prepare the skin

Make sure the periwound skin is clean, dry and intact.

- Use Skintegrity Wound Cleaner (MSC6008EP) to clean the wound
- Use SurePrep Rapid Dry (MSC1610) to protect the skin (optional)



Step 2: Measure the tape

Unroll desired length of tape and cut or tear.



Step 3: Apply the tape

Stick tape around the dressing/tubing and apply firm pressure. Try to avoid gaps and wrinkles that might allow moisture to get between the tape and skin.



Step 4: Check for comfort

Ensure the tape is snug, but not too tight, as excessive pressure may cause skin irritation or restrict movement.



Step 5: Remove the tape

Gently peel back the tape starting at the edge. Remove it slowly in the direction of hair growth to minimize discomfort and skin damage. If tape remains, use a medical adhesive remover to help loosen.

- Use SurePrep Adhesive Remover Spray (MSC1651) to help loosen

Medline Industries, LP | Three Lakes Drive, Northfield, IL 60093 | 1-800-MEDLINE (633-5463)

© 2024 Medline Industries, LP. All rights reserved. Medline is a registered trademark of Medline Industries, LP.
MKT19W6453914 / e24628 / 92

Learn more

Contact your Medline Representative or visit [medline.com](https://www.medline.com)