

# bolink<sup>®</sup>


## D Cap Gravity Feeding Set with ENFit<sup>®</sup> Connector

**REF** BK - 2260



**Indications for Use:** The U Deliver Bolink<sup>®</sup> ENFit<sup>®</sup> Enteral Feeding Sets are intended for over-the-counter use to deliver liquid nutritional formulas or water to a patient's enteral access device (feeding tube).

**Device Description:** The Bolink<sup>®</sup> D Cap gravity feeding set consists of tubing (A) with a roller clamp (B), a screw cap (C), an ENFit<sup>®</sup> connector (D) for connecting to feeding tubes and a protective cap (E) and vent (F).

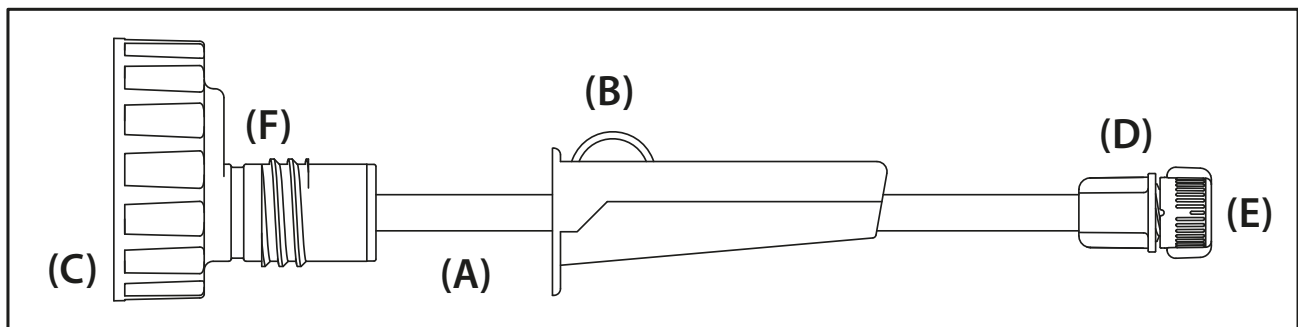

The Bolink<sup>®</sup> D Cap gravity feeding set is intended to connect compatible commercially available food containers with a 26 mm diameter opening to feeding tubes with ENFit<sup>®</sup> proximal connectors. Not made with natural rubber latex. Not formulated with DEHP or BPA.

### Warnings:

- For enteral feeding only.
- This device incorporates an ENFit<sup>®</sup> Connector which is designed to reduce the likelihood of tubing misconnections; however, the potential to misconnect this device with connectors of other healthcare applications still exists.
- Small parts are a potential choking hazard if this device is not used as intended. Use under adult supervision.

### Precautions:


- This device should be discarded after 24 hours and replaced with a new device.
- For single patient use only.
- Dispose with household waste. Do not recycle.
- Use with food containers holding greater than 500 mLs requires supervision and flow management using the roller clamp.

**Instructions for Use:**

1. Confirm feeding tube position and flush following manufacturer's instructions.
2. Close roller clamp **(B)** on Bolink® Tubing **(A)**.
3. Remove cap on compatible food container, attach screw cap **(C)** to filled food container and confirm cap is secured tightly.
4. Open roller clamp **(B)** and turn food container upside down to fill Bolink® tubing **(A)**.
5. When tubing is filled, close roller clamp **(B)**.

**WARNING:** Before connecting to a feeding tube, confirm that the Bolink® tubing is being connected to a feeding tube only. DO NOT connect to non-feeding tube. This includes confirming that any transitional connector with a Christmas tree tip is connected to a feeding tube and not IV tubing.

6. Remove protective cover **(E)**, use ENFit® connector **(D)** to attach Bolink® tubing to feeding tube. If the feeding tube has a funnel instead of an ENFit® connector use a transition connector with Christmas tree tip.
7. Open clamp **(B)**, hold food container above feeding tube to start gravity flow, and, if necessary, adjust roller clamp to control delivery rate.
8. When finished, close roller clamp **(B)**, disconnect Bolink® tubing **(A)** from feeding tube and food container.
9. Flush Bolink® tubing **(A)** with tap water by holding under a faucet and letting water flow through the tubing until the water runs clear. While flushing, cover the vent **(F)** opening with your finger to keep the vent membrane's outside surface dry and ensure air flow necessary for venting in future uses.
10. Flush feeding tube following manufacturer's directions.

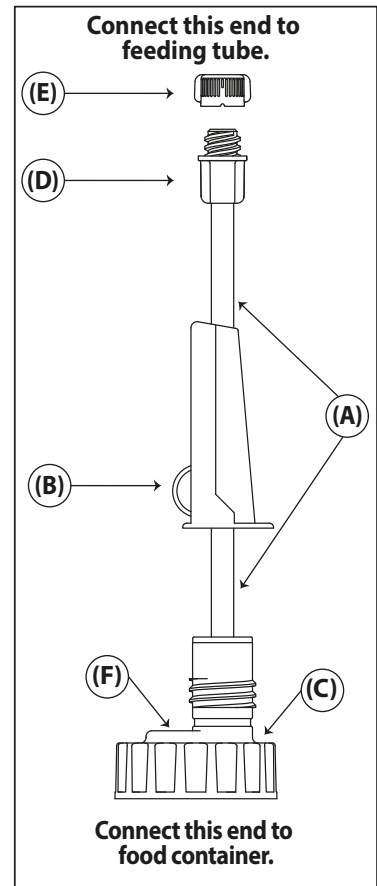


**Symbol Glossary:**

Catalogue number		Use-by date	
Indicates the manufacturer's catalogue number so that the medical device can be identified. Ref. 5.1.6 in ISO 15223-1:2016		Indicate the date after which the medical device is not to be used. Ref. 5.1.4 in ISO 15223-1:2016	
<b>Non sterile.</b> Indicates a medical device that has not been subjected to a sterilization process. Ref. 5.2.7 in ISO 15223-1:2016		<b>Consult instructions for use</b> Indicates the need for the user to consult the instruction for use. Ref. 5.4.3 in ISO 15223-1:2016	
<b>Keep Dry.</b> Indicates a medical device that needs to be protected from moisture. Ref. 5.3.4 in ISO 15223-1:2016		<b>Do not use if package is damaged</b> Indicates a medical device that should not be used if the package has been damaged or opened. Ref. 5.2.8 in ISO 15223-1:2016	
<b>Do not reuse</b> Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure. Ref. 5.4.2 in ISO 15223-1:2016		<b>Manufacturer</b> Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC. Ref. 5.1.1 in ISO 15223-1:2016	
<b>Batch code</b> Indicates the manufacturer's batch code so that the batch or lot can be identified. Ref. 5.1.5 in ISO 15223-1:2016			

Note: Symbols are contained in ISO 15223-1:2016 Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1: General requirements

For more information on Bolink® tubing and U Deliver Medical products, visit [www.UDeliverMedical.com](http://www.UDeliverMedical.com).

 U Deliver Medical, LLC.  
7900 Excelsior Blvd, Suite 108  
Hopkins, MN 55343  
Made in Italy

Bolink® is a trademark of U Deliver Medical. ENFit® is a trademark of GEDSA, used with permission. ©2020 U Deliver Medical, LLC