

Anti-embolism stockings

McKesson anti-embolism stockings can help reduce the threat of blood clots forming in legs. Blood clots can form in anyone, but the threat of their formation is much greater during a hospital stay. McKesson anti-embolism stockings improve blood circulation in the leg veins by applying graduated compression. The stockings are designed for easy application and comfort while wearing. They are made of nylon/spandex with a graduated compression, inspection toe hole and a formed heel pocket. Color-coded for easy sizing.

Indications:

- To help prevent formation of deep vein thrombosis.
- To promote increased blood flow velocity in the legs by compression of the deep venous system.

Contraindications:

Stockings are not recommended for patients with the following:

- Any local leg condition in which stockings would interfere, such as dermatitis, immediate post-operative vein ligation, gangrene or recent skin graft.
- Severe arteriosclerosis or other ischemic vascular disease.
- Massive edema (swelling) of legs or pulmonary edema from congestive heart failure.
- Extreme deformity of leg.



Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Never Compromise on CareSM

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates (in McKesson in). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.


Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

ORDERING INFORMATION

MFR #	DESCRIPTION	QTY
84-01M	Anti-Embolism Stockings, Knee-High, Regular, Small	1 pair/pack, 12 packs/cs
84-02M	Anti-Embolism Stockings, Knee-High, Regular, Medium	1 pair/pack, 12 packs/cs
84-03M	Anti-Embolism Stockings, Knee-High, Regular, Large	1 pair/pack, 12 packs/cs
84-04M	Anti-Embolism Stockings, Knee-High, Regular, X-Large	1 pair/pack, 12 packs/cs
84-11M	Anti-Embolism Stockings, Knee-High, Long, Small	1 pair/pack, 12 packs/cs
84-12M	Anti-Embolism Stockings, Knee-High, Long, Medium	1 pair/pack, 12 packs/cs
84-13M	Anti-Embolism Stockings, Knee-High, Long, Large	1 pair/pack, 12 packs/cs
84-14M	Anti-Embolism Stockings, Knee-High, Long, X-Large	1 pair/pack, 12 packs/cs
84-21M	Anti-Embolism Stockings, Thigh-High, Short, Small	1 pair/pack, 12 packs/cs
84-22M	Anti-Embolism Stockings, Thigh-High, Regular, Small	1 pair/pack, 12 packs/cs
84-23M	Anti-Embolism Stockings, Thigh-High, Long, Small	1 pair/pack, 12 packs/cs
84-31M	Anti-Embolism Stockings, Thigh-High, Short, Medium	1 pair/pack, 12 packs/cs
84-32M	Anti-Embolism Stockings, Thigh-High, Regular, Medium	1 pair/pack, 12 packs/cs
84-33M	Anti-Embolism Stockings, Thigh-High, Long, Medium	1 pair/pack, 12 packs/cs
84-41M	Anti-Embolism Stockings, Thigh-High, Short, Large	1 pair/pack, 12 packs/cs
84-42M	Anti-Embolism Stockings, Thigh-High, Regular, Large	1 pair/pack, 12 packs/cs
84-43M	Anti-Embolism Stockings, Thigh-High, Long, Large	1 pair/pack, 12 packs/cs
84-44M	Anti-Embolism Stockings, Thigh-High, Regular, X-Large	1 pair/pack, 12 packs/cs
84-45M	Anti-Embolism Stockings, Thigh-High, Long, X-Large	1 pair/pack, 12 packs/cs

SIZING GUIDE

		MFR #	SIZE	CALF CIRCUMFERENCE	LENGTH	TOP COLOR	TOE PORT COLOR
KNEE	REGULAR	84-01M	Small	Up to 12 in	Up to 16 in	White	Yellow
		84-02M	Medium	12 in to 15 in	16 in to 17 in	White	White
		84-03M	Large	15 in to 17 in	17 in to 18 in	White	Blue
		84-04M	X-Large	17 in to 19 in	18 in to 19 in	White	Green
	LONG	84-11M	Small	Up to 12 in	Up to 18 in	Blue	Yellow
		84-12M	Medium	12 in to 15 in	18 in to 19 in	Blue	White
		84-13M	Large	15 in to 17 in	19 in to 20 in	Blue	Blue
		84-14M	X-Large	17 in to 19 in	20 in to 21 in	Blue	Green
THIGH	SHORT	84-21M	Small	Up to 12 in	Up to 29 in	Yellow	Yellow
		84-31M	Medium	12 in to 15 in	Up to 29 in	Yellow	White
		84-41M	Large	15 in to 17 in	Up to 29 in	Yellow	Blue
	REGULAR	84-22M	Small	Up to 12 in	29 in to 33 in	White	Yellow
		84-32M	Medium	12 in to 15 in	29 in to 33 in	White	White
		84-42M	Large	15 in to 17 in	29 in to 33 in	White	Blue
		84-44M	X-Large	17 in to 19 in	29 in to 33 in	White	Green
	LONG	84-23M	Small	Up to 12 in	33 in to 36 in	Blue	Yellow
		84-33M	Medium	12 in to 15 in	33 in to 36 in	Blue	White
		84-43M	Large	15 in to 17 in	33 in to 36 in	Blue	Blue
		84-45M	X-Large	17 in to 19 in	33 in to 36 in	Blue	Green

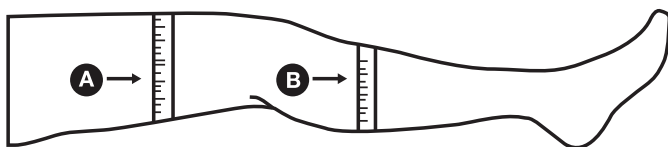

FOR PROPER FIT – KNEE LENGTH

Size based on: Circumference of “B”
Length based on: Distance from bend of knee to bottom of heel

FOR PROPER FIT – THIGH LENGTH

Size based on: Circumference of “A” and “B”
Length based on: Distance from “A” to bottom of heel

LAUNDERING GUIDELINES

Machine wash in lukewarm water. Air dry/tumble dry on low setting. Do not iron.