



270 WASHINGTON STREET
MOUNT VERNON, NY 10553
www.proactivemedical.com
855-237-7622

Sit-to-Stand Transfer sling Instruction manual

- ⚠ WARNING:** Important! Read and understand these instructions before assembling or using the sling. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the sling is not properly applied, personal injury and damage to the sling could result.
- ⚠ WARNING:** If components are damaged or missing, contact your Proactive Medical authorized distributor immediately.
- ⚠ WARNING:** Carefully inspect the sling before each use for wear and damage to seams, fabric, straps, and strap loops. Torn, cut, frayed or broken slings can fail, resulting in serious personal injury to the user. Use only slings that are in good condition. Discard and destroy old, unusable slings.
- ⚠ WARNING:** DO NOT exceed the sling's maximum weight capacity.
- ⚠ WARNING:** Notice for California Customers- California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.
- ⚠ WARNING:** Proactive Medical Products assumes no responsibility for any damage or injury caused by improper application or use of this product.

NOTE: For this procedure, refer to **FIGURE 1.0** and **FIGURE 1.1**

POSITIONING THE SLING

1. Review the following procedures carefully before attempting lift; call Proactive Medical’s Technical Support at 845-205-6004 with any questions.
2. Determine the best attachment point for the sling on each lift arm. Use the following chart as a guideline to determine attachment point. Refer to Detail “A”.

NOTE: The following chart is for reference ONLY.

- ⚠ WARNING:** ALWAYS use the color coded strap on the sling closest to the patient while still maintaining patient stability and comfort. Make sure the color coded strap sits flush against the attachment knob in Detail “A” of **FIGURE 1.1** on page 3, otherwise injury or damage can result.

NOTE: The color coded straps can be used to place the patient in various positions. The colors make it easy to connect both sides of the sling equally.

3. Pull the thigh supports forward and attach to the lower attachment points just above the pivoting knee pad.
4. Attach the top portion of the sling to each lift arm at the attachment point determined in STEP 1.

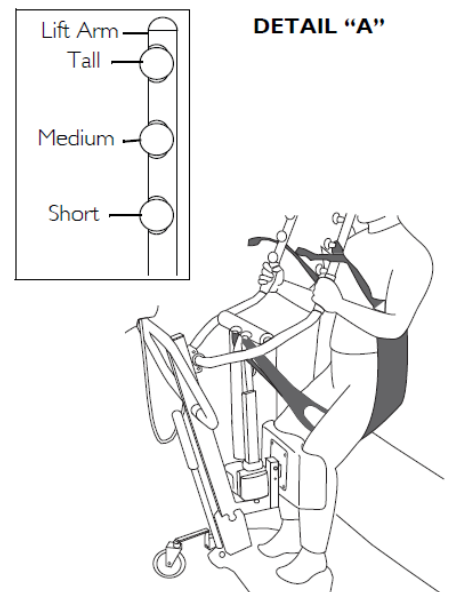


FIGURE 1.0 Attaching Slings to the Stand Up Lift - Transport Slings

TRANSFER SLING:

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

5. Make sure of the following:

- The sling is wrapped around the outside of each lift arm.
- The closest color coded strap to the patient is used to maintain stability and comfort.
- The colored coded strap sits flush against the attachment knob.
- The same top color coded strap is used on each side of the sling.
- The same attachment point is used on each lift arm.
- The same bottom color coded strap is used on each side of the sling. Refer to Detail “B” of FIGURE 1.1.

NOTE: The top and bottom colored straps do not have to be the same.

NOTE: If necessary, adjust the attachment point of the sling to each lift arm.

6. Once a stable and comfortable position is found for the patient, mark the attachment points and color coded strap used in the patient file for future use.

7. Transfer patient. Refer to Transferring a Patient in the Stand Up Lift Owner's Manual.

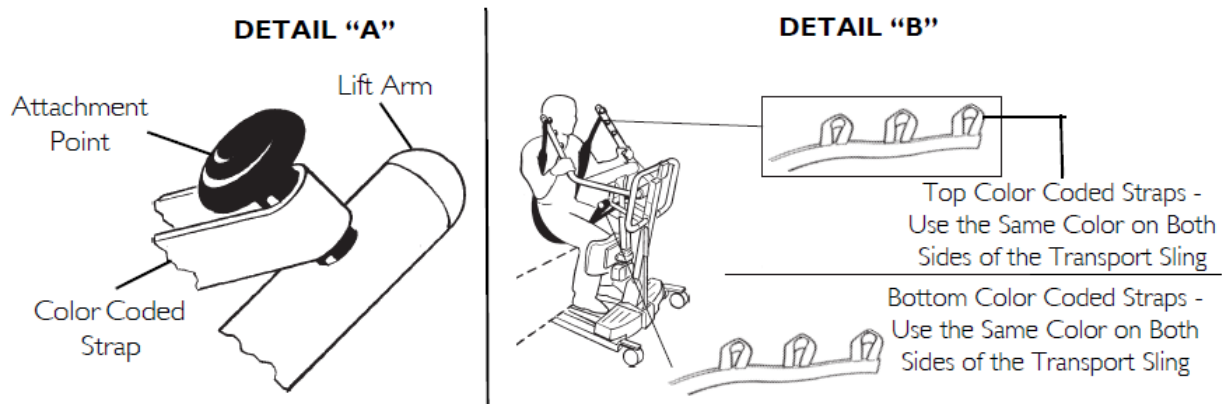


FIGURE 1.1 Attaching Slings to the Stand Up Lift - Transport or Transfer Slings



TRANSFER SLING:

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

Washing Instructions:

1. Machine wash warm or cold
 - a. Maximum washing temperature 185°F (85°C)
 - b. Wash at 160°F (71°C) for 3 minutes
 - c. Wash at 145°F (63°C) for 10 minutes
 - d. Air dry, or tumble dry at cool or very low temperature
 - e. Do not tumble dry at high temperature
2. Use non-enzyme detergent
3. Do not use bleach
4. Do not wash with other colors

WARNING: After each laundering (in accordance with instructions on the sling), inspect sling(s) for wear, tears, and loose stitching. Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard immediately.

LIMITED LIFETIME WARRANTY:

Your Proactive Medical sling product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser. Useful life of this product is six months from date of purchase under normal use, however, heavy use or excessive washing may reduce the useful life of the product. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

Sling Material: Polyester / Straps Material: Polypropylene