www.drivemedical.com

REV1.4.14.14

Home Bed Assist Handle
Mango Auxiliar Para La
Cama

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Item # RTL15073


www.drivemedical.com

REV1.4.14.14


Installation Instructions

The Home Bed Assist Handle is designed for use with most mattress and box spring type beds.

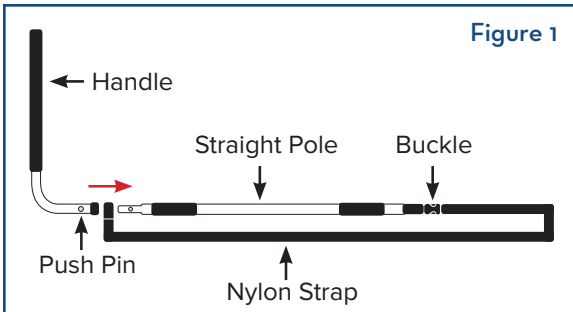
1. Remove contents from carton.
2. Slide end of nylon strap over opposite end of the straight pole, leaving room for the handle to be installed. Repeat for the second pole.
3. Attach handle section over the end of the straight pole by depressing brass push pin and sliding into place, making sure push pin snaps back into place. This will secure the strap to the bed assist handle (Figure 1).

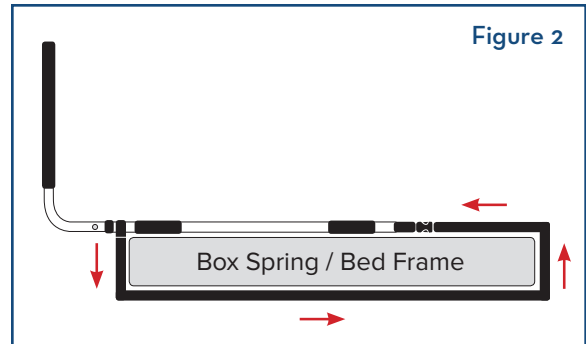4. Remove mattress from the bed.
5. Position safety rail at desired position. The space between the vertical handle bars and other bedside device or bed board must be less than 2.25 inches (60mm), or larger than 9.25 inch (235mm).
6. Unbuckle strap. Wrap strap around the underside of the box spring / bed frame and re-buckle to secure in place. Tighten strap as necessary (Figure 2).

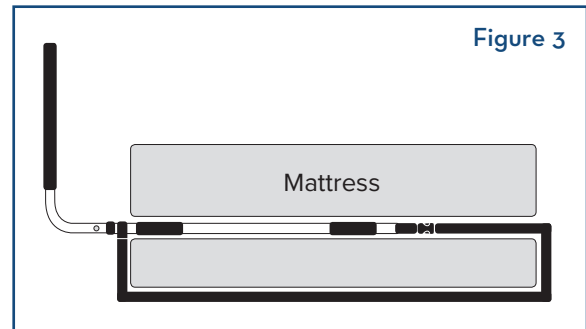7. Return mattress to bed (Figure 3).

WARNING

- This assist handle is intended for assisting in and out of bed only. DO NOT use for other purposes.
- Mattress must fit bed frame and assist handle snugly to prevent potential patient entrapment.
- Follow installation instruction to prevent potential patient entrapment.
- Patient entrapment may cause injury or death!

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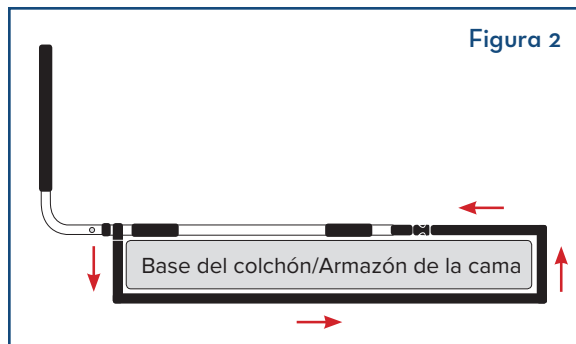
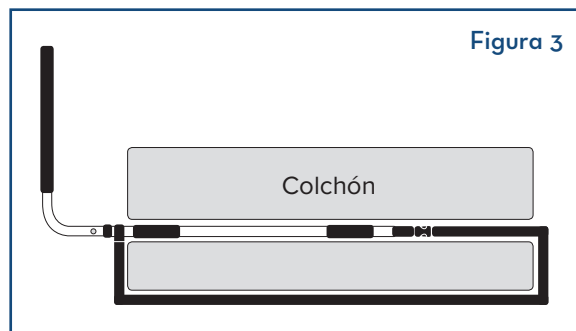
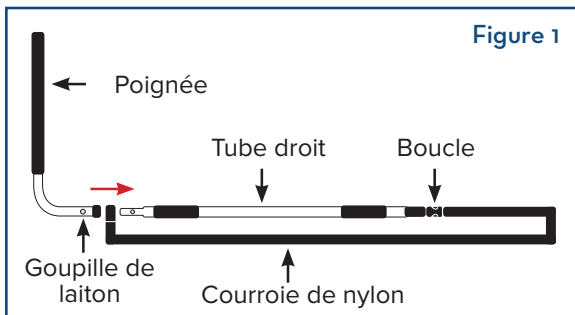
lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2013 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in USA

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© 2013 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2013 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.