

Be Independent.™

INDEPENDENCE BED TABLE™

BECAUSE YOUR BED RAIL CAN PROVIDE SAFETY AND CONVENIENCE


NEW
FALL 2012



PRODUCT #5900



WATCH A DEMO

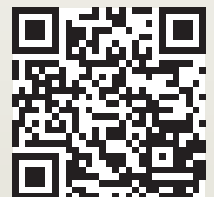



Ergonomic Grip
Assists In and Out of Bed



Tray Swivels Out of the Way



Includes Organizer Pouch


FEATURES & BENEFITS

- **Ergonomic Handle** - Allows for easy transfer in and out of bed.
- **Swivel Tray** - Multi-Use laptop/TV tray. Tray swivels 360°.
- **A Pocket Organizer** - Provides storage space for handy items.
- **Extendable Legs** - Provides added stability when standing up.
- **Height Adjustment** - Accommodates any home or hospital bed.
- **Easy Installation** - Installs in seconds with no additional tools required.

SPECIFICATIONS

Height Adjustment From Floor to Top of Handle: 21" - 44"
 Height Adjustment From Mattress Base to Top of Handle: 17.5" - 22.5"
 Rail Width: 19"
 Height Range From the Floor to Horizontal Support Structure Under Mattress: 13.5" - 21.5"
 Depth of Horizontal Support Structure Under Mattress: 25"
 Weight Capacity of Rail: 300 lbs
 Weight Capacity of Tray: 30 lbs
 Weight of Product: 10.5 lbs
 Made of: Steel frame with durable powder coat finish
 Package Dimensions: 28" L x 20.25" W x 2.5" H; 9 lbs
 Case of 4 Dimensions: 28" L x 20.25" W x 10" H; 36 lbs
 Size of Tray: 20" x 17"


EASY ASSEMBLY (No Additional Tools Required)

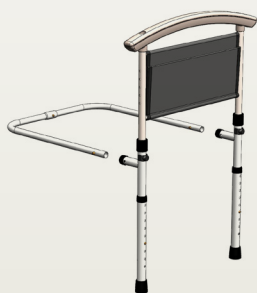

Attach Base Tubes into the Base Support Tubes. Make Sure the Spring Pins Engage.



Insert the Tray Pivot Tube into One of the Desired Tray Pivot Tube Receiver Holes Located on Top of the Handle.

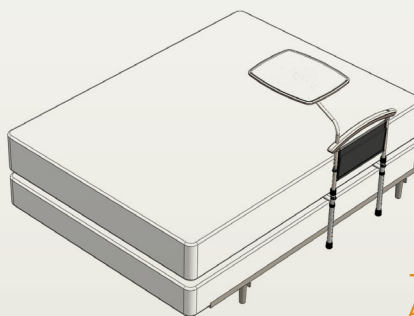

Fully Insert the Assembled Product Between the Mattress and Box Spring of the Bed.



The Height of the Tray Can Be Adjusted by Removing the Tray Pivot Tube From the Handle. Adjust the Clip Pins by Removing them and Reinserting the Clip Pins in the Holes Located in the Handle Upright Tubes.