



TENA® Youth Briefs

For Moderate to Heavy Bladder and/or Bowel Leakage Protection...

Now featuring a soft non-woven backsheet and hook & loop fastening. TENA® Youth Brief's new cloth-like feel will provide the wearer with more comfort and dignity.

NEW & IMPROVED



	Light			Medium			Heavy			
TENA®	3 droplets	4 droplets	5 droplets	6 droplets	7 droplets	8 droplets	9 droplets	10 droplets	11 droplets	12 droplets
Youth Brief										



For the User:
A comfortable, secure and discreet solution.



For the Carer:
Easy to apply and change.



For the Purchaser:
Quality product at an affordable price.

Feature

- InstaDri Skin-Caring System™
- Dry-Fast Dual Core™
- Super-absorbent Polymer
- Hook & Loop Fastening
- Wetness Indicator

Benefit


- Improves surface dryness, promoting healthier skin. Transfer layer permits fluid to almost instantly enter the dual core.
- Rapidly wicks fluid away from skin.
- Improves fluid retention.
- Delivers unlimited and secure fastenability.
- Alerts the caregiver that the brief has been soiled and needs to be changed.

Youth Brief



TENA® Youth Brief Fitting Guide

Lying

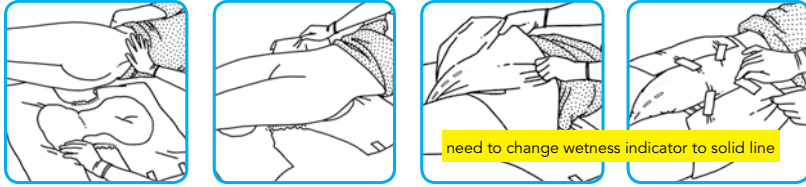

1. Lying: Roll person onto his/her side. Place brief with the flap folded under nearest to body.

Sitting: Open brief and place plastic cover against the back of the chair.

Standing: While person is standing have them hold onto something for support.

Sitting



2. Lying: Roll resident back and unfold flap.

Sitting: Position person onto the brief.

Standing: Place a clean brief between the legs from front to back.


3. Pull brief firmly, pulling out, then up. Brief should fit snugly in the groin. Smooth the front panel.

4. Fasten bottom hook tabs first, angling up, then top hook tabs pointing down for best fit. Press the hook tab into the cloth-like backcover. Give a slight tug to anchor the hooks in the backcover fibers.

5. Adjust the brief by pulling the folds of the inner thigh down and fitting the brief into the groin.

6. Adjust the outer garments.

Standing

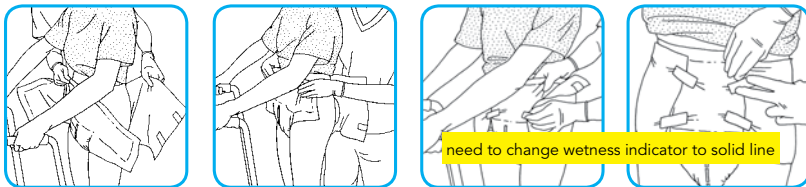

TENA® Youth Brief

Product Name	Order #	Hip Size	Color	Case Count	Format
TENA® Youth Brief	61199	17" - 29"	White	90	3/30

People who bought TENA® Youth Brief also bought...



TENA® Small Briefs

For moderate to heavy bladder and/or bowel incontinence. Designed to meet the needs of smaller individuals.



TENA® Classic Washcloths

Gentle cleansing that offers effective and economic personal hygiene, ideal for those with healthy, yet fragile skin.



TENA® Cleansing Cream

3-in-1 formula promotes skin health by gently cleansing, moisturizing and soothing vulnerable skin.

SCA Hygiene Support



TENA® Advice Line:
1.866.722.6659



info.usa@tena.com



www.TENA.us



TENA® Briefs

Together we make a difference



SCA Care of life. Because our products make life easier for you and for millions of people around the world. Because our resources and the way we work are natural parts of the global lifecycle. And because we care.

