

one year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*iteRe/hab

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

trotter canopy

Item # TR 8026



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

*Wenzel*iteRe/hab

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING



installation

1. Adjust canopy width to fit your Trotter chair by pulling apart or pushing together canopy frame.
2. Insert canopy rods (A) into sockets (B) on the back of the Trotter chair.
3. Place rubber collars at desired height and push canopy down until the rubber stoppers reach the socket.
4. Remove screws from underside of three curved canopy supports (C) and adjust width.
5. Replace screws and tighten.



adjustments

To adjust height, raise or lower the rubber collars on both canopy rods to desired height.