

# Limited Warranty

Congratulations on your purchase of your Standers brand product. We hope that it will ease your daily living.

Standers, Inc. warrants to the original purchaser that this product will be free from defects in material and/or workmanship for one (1) year from the date of purchase. A receipt of purchase showing the purchase date is required for proof of purchase. During the warranty period, Standers, Inc. will correct any defects in material or workmanship, at no charge. The original purchaser, however, must prepay shipping cost. Any replacement parts will be new and are warranted for the remainder of the original warranty or thirty (30) days from the date of shipment, whichever is longer. The original purchaser must notify Standers, Inc. during the warranty period in writing of the defect in workmanship or material. This warranty is not transferable and covers only normal use. NO SHIPPING CHARGES ARE REFUNDABLE.

The warranty does NOT cover the following:

- Damage during shipment other than original shipment to retailer.
- Damage caused by unauthorized attachments, alterations, or modifications.
- Damage caused by any other abuse, misuse, mishandling or misapplication.
- Damage caused by the use of the product for purposes other than those for which the product was designed or intended.

Under no circumstances shall Standers, Inc. be liable for any special, incidental, or consequential damages based upon breach of this limited warranty, breach of contract, or strict liability.

THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THE DESCRIPTION ON THE FACE HEREOF. ALL OTHER WARRANTIES, EXPRESS OR IMPLIED (INCLUDING ALL IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE), ARE HEREBY DISCLAIMED.

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal right, and you may also have other rights, which vary from state to state. To obtain service under this warranty, you can reach Standers Inc. at (800) 506-9901 or (435) 755-0453.

## **WARNING: Please read before using this product.**

1. Serious injury or death can result if bed rails are not properly assembled and installed. Read the enclosed Entrapment Prevention Guidelines, or if your product did not come with a copy of the Entrapment Prevention Guidelines, visit the stander.com website to review these guidelines or call Stander at (435) 755-0453 for a copy.
2. Entrapment has resulted in injuries and death. Entrapment is a situation where an individual can become wedged between a bed rail and any other object. The possibility of an individual's head, body, arms, or legs could be trapped between the bars of a bed rail, between the bed rail and a head or footboard of a bed, or between the bed rail and a mattress. Standers Inc. is committed to providing safe products and informing the user of ways to help prevent entrapment.
3. Individuals who have problems with memory, sleeping, incontinence, pain, uncontrolled body movement, who get out of bed and walk unsafely without assistance, frail, weak, confused, restless, under the influence of drugs or any substance that could affect their judgment, are susceptible to entrapment.
4. Individuals with size or weight conditions need to consider the possible increase risk of entrapment due to personal mobility, mattress deformation, and the gaps that can be created between the bed rail and bed.
5. Stander Incorporated includes safety devices on certain bed rails to help prevent entrapment. Typical safety devices are sewn bed rail covers and safety straps. These safety devices must always be installed properly and must not be taken off the product at any time; failure to use these safety devices will increase the risk of injury or death.
6. The product was not intended to carry the full weight of an individual. This product is only intended to provide balance and support while sitting and standing.
7. Children should not be allowed to hang on or play with the product.
8. Bed rails should not be installed on beds with light weight foam, air, or other lightweight mattresses. These light weight mattresses can deform or allow the bed rail to move and create gaps that will increase the risk of entrapment.
9. Bed rails should not be installed on water beds since the mattress can deform to create gaps that will increase the risk of entrapment.
10. Bed rails should not be used on adjustable frame beds since the mattress can move relative to the bed rail and create gap that will increase the risk of entrapment.
11. **DISCLAIMER:** Those who install a Stander Inc. bed rail and those using the bed rail agree to read the Entrapment Prevention Guidelines and must make their own assessment of installing Stander Inc. bed rails onto their beds and assume the risk of using Stander Inc. bed rails. Neither Standers, Inc. nor the manufacture is liable for any harm, injury or death that may result or has resulted from using Stander Inc. bed rails.

## **SAFETY PRECAUTIONS:**

- ✓ Regularly check the bed rail to identify the seven zones of possible entrapment (see Entrapment Prevention Guidelines). Bed rails will likely move over time with use.
- ✓ Never remove sewn bed rail covers, safety straps, or any other safety device to help prevent entrapment.
- ✓ Avoid open spaces between the perimeter of the bed rail. See Entrapment Prevention Guidelines zone 1.
- ✓ Avoid gaps under the bed rail and between the top of the mattress. See Entrapment Prevention Guidelines zone 2.
- ✓ Avoid gaps between the bed rail and the side of the mattress. See Entrapment Prevention Guidelines zone 3.
- ✓ Avoid gaps under the bed rail and at the ends of the bed rail. See Entrapment Prevention Guidelines zone 4.
- ✓ Avoid using 2 bed rails on the same side of the bed. See Entrapment Prevention Guidelines zone 5
- ✓ Avoid gaps between bed rails and head or footboards that would allow entrapment of the head, neck, or chest. See Entrapment Prevention Guidelines zone 6.
- ✓ Avoid gaps between the head or foot board of a bed and the end of the mattress. See Entrapment Prevention Guidelines zone 7.
- ✓ If you have any concerns or questions concerning the Warning and/or Safety precautions, please contact Standers at (800) 506-9901 or (435) 755-0453.




EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands  
Tel: +31 (0)70 345 8570  
Fax: +31 (0)70 346 7299




# Assembly Instructions

1. Remove all parts from the box and identify the parts as shown in Figure 1.

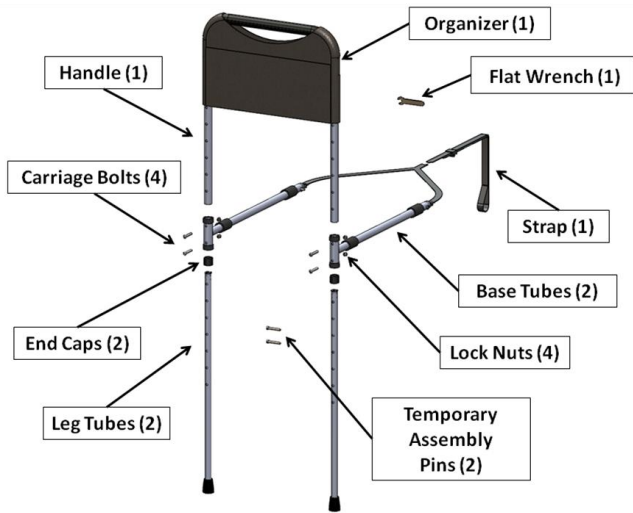


**FIGURE 1**

2. Slide Base Tubes onto the Handle as shown in Figure 2. After sliding Base Tubes onto the Handle, secure the End Caps onto the End of Handle Tubes.

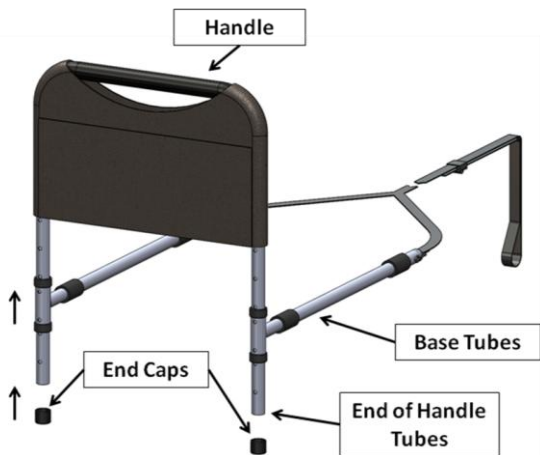

**FIGURE 2**

3. Insert Leg Tubes into End of Handle Tubes so that the Bottom Holes in the Leg Tubes align with The Bottom Holes in the Handle Tube. With the holes aligned, insert the Temporary Assembly Pins as shown in Figure 3.

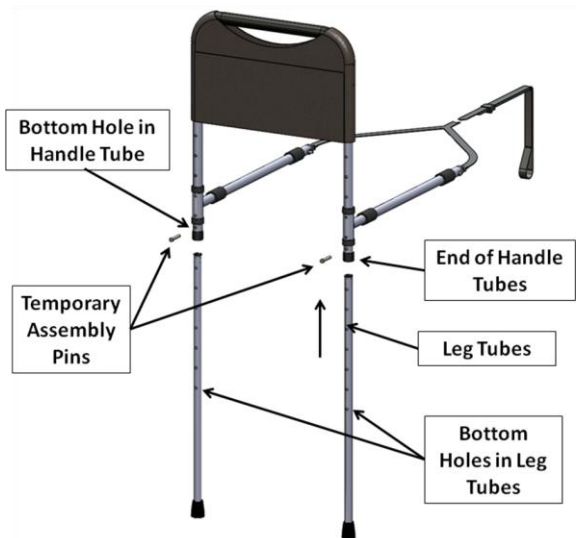


**FIGURE 3**

**IMPORTANT:** Before proceeding, please read the "Guidelines to Prevent Entrapment" for proper installation.

4. Place Base Tubes between the mattress and box spring of the bed. Remove the Temporary Assembly Pins and adjust the Leg Tubes so that the Leg Tubes engage the floor AND adjust the Top of


the Handle Tube to a safe height relative to the Top of the Mattress. Realign the holes in the Leg Tubes with the Bottom holes in the Handle Tubes and insert the Temporary Assembly Pins as shown in Figure 4.

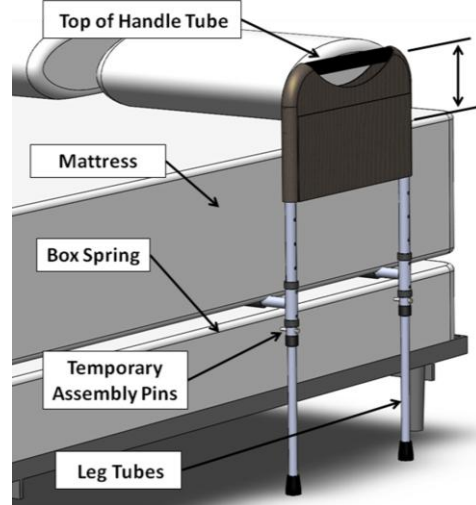

**FIGURE 4**


5. Using the Flat Wrench, secure the Handle Tubes to the Base Tubes with the Carriage Bolts and Lock Nuts as shown in Figure 5.

**NOTE:** Make sure the Leg Tubes engage the floor before installing the Carriage Bolts and Lock Nuts.

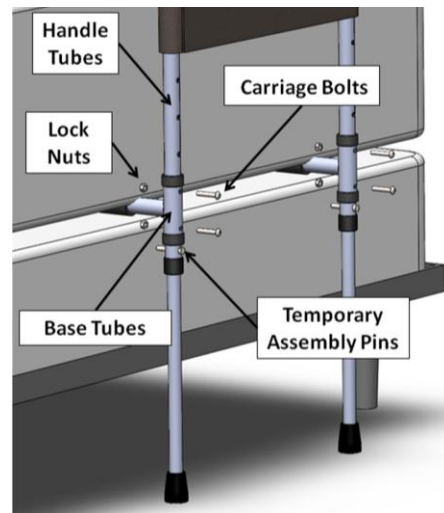

**FIGURE 5**

6. Slide Bed Rail against the side of the mattress and tighten the Strap to the Bed Frame to prevent the bed rail from moving away from the bed as show in Figure 6.

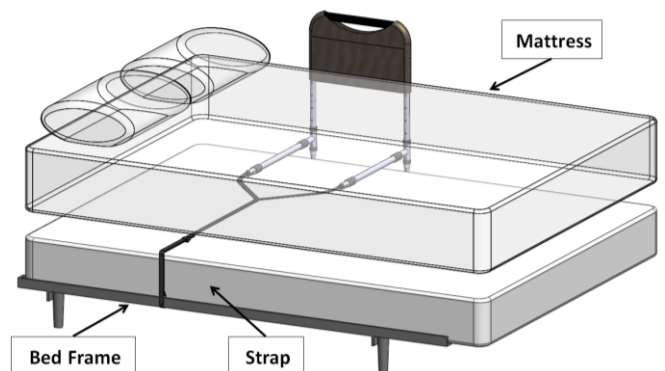

**FIGURE 6**

**IMPORTANT:** The Organizer and Strap are for the user's safety. Do not remove these items. Removing these items will put the user of the bed rail at risk of entrapment.