


# Attends®


## Get the Attends® Advantage

### BRIEFS SIZING GUIDES

#### Attends® & Attends® Care Briefs (DDC, DDA, DDEW, DDP, BRBX15 & DD50)

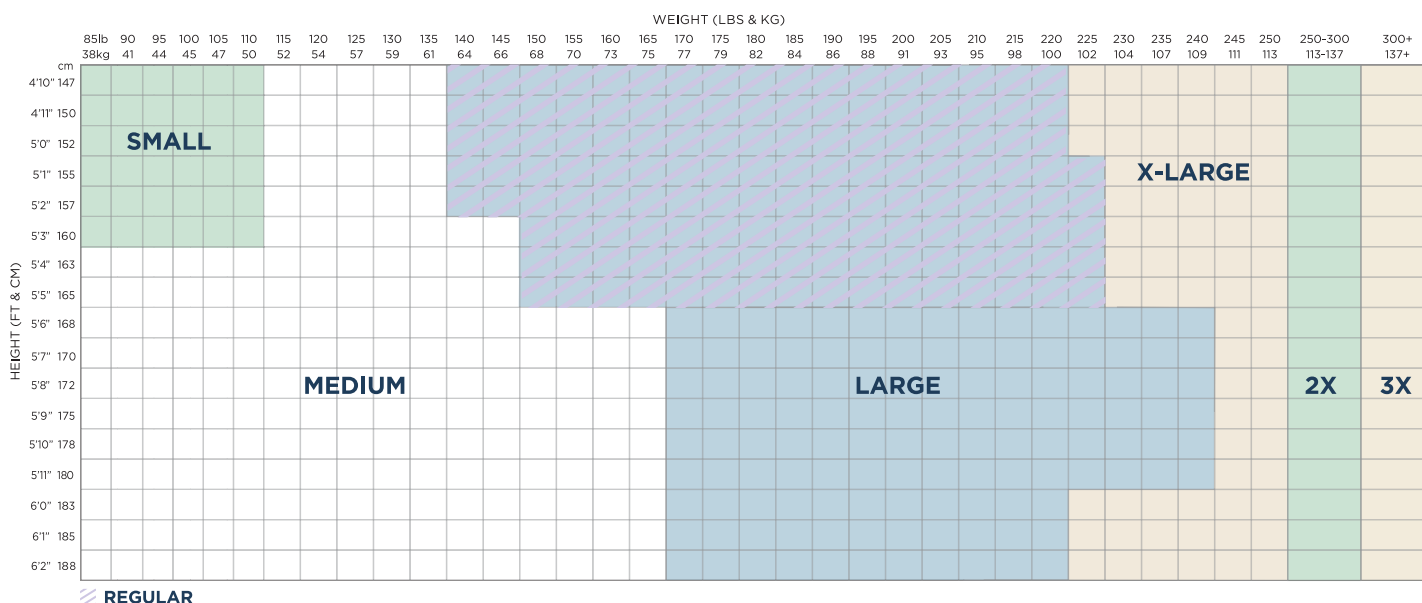

#### Attends® Stretch Briefs (DDS & DD60)

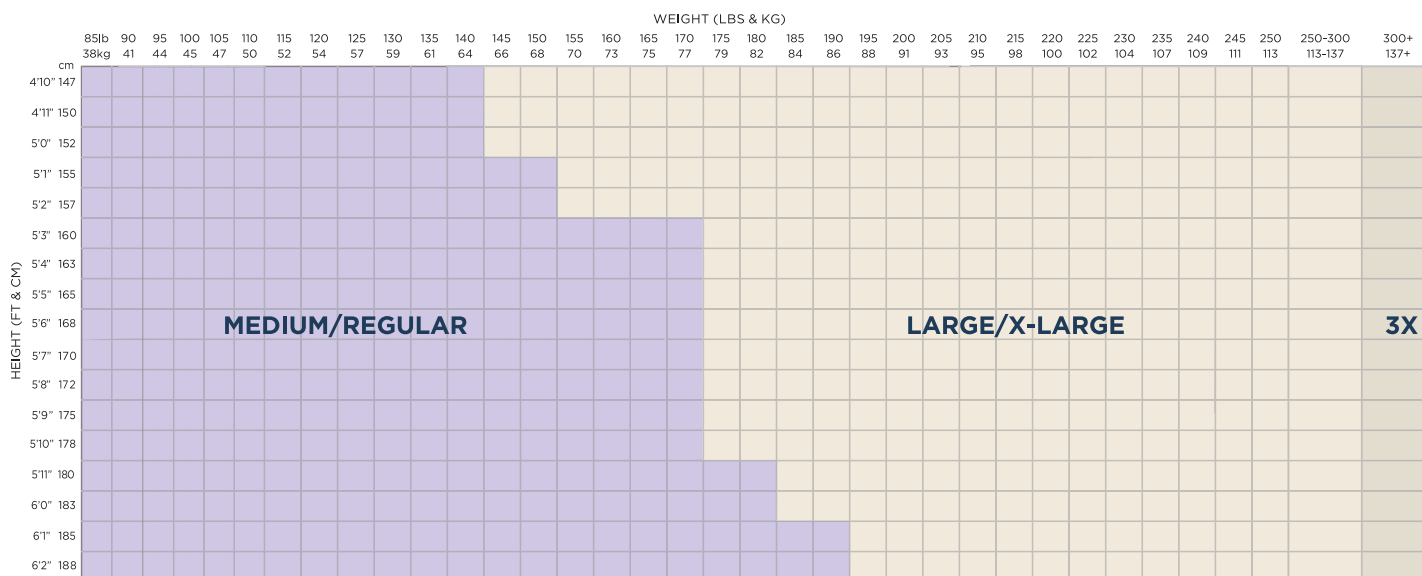

# Attends®

1.800.428.8363 • [www.attends.com](http://www.attends.com)


© 2018 Attends Healthcare Products, Inc. 45740 / V1 / 3.2018

**Domtar**  
personal care

# Attends®

## Get the Attends® Advantage

### HOW TO APPLY ATTENDS® BRIEFS

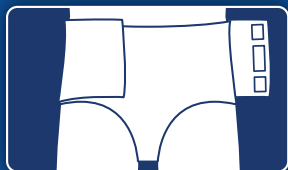

#### Standing Application

1. Fully open brief. Fold in half lengthwise. Place between legs, with inner lining against skin.
2. Open and lift back of brief so top of wetness indicator aligns with coccyx. Pull front of brief up between legs, with pad snug against perineum. Smooth side panels across hips. Front/back of brief should be even at the waist. Gently move excess skin while positioning leg gathers in the natural crease of the groin.
3. Fasten tape tabs. Adjust panels as necessary for a comfortable, secure fit.



#### In-Bed\* (Steps 1, 2 & 6) / Seated Application\*\* (Steps 3, 4, 5 & 6)

- \*1. While the individual is on the bed fully open brief. Fold in half length wise. Place between legs, with inner lining against skin.
- \*2. Open and lift back of brief so top of wetness indicator aligns with coccyx. Pull front of brief up between legs, with pad snug against perineum. Smooth side panels across hips. Front/back of brief should be even at the waist. Gently move excess skin while positioning leg gathers in the natural crease of the groin.
- \*\*3. If individual is in a seated position help them stand, making sure they are stable.
- \*\*4. Place brief on seat, so they will sit in brief center. Position soft side against skin.
- \*\*5. Seat individual on brief. Align the top of the wetness indicator to the coccyx.
6. Fasten tape tabs. Adjust panels as necessary for a comfortable, secure fit.

Attends®

1.800.428.8363 • [www.attends.com](http://www.attends.com)  
© 2018 Attends Healthcare Products, Inc. 45740 / V1 / 3.2018

Domtar®  
personal care