FRONT

BACK

FIBER CONTENTS OF COVER: 100% Polyester MADE IN CHINA /
CONTENTS OF FILLING: Buckwheat FABRICADO EN CHINA

BED BUDDY®**Back Wrap**

CAUTION: READ BEFORE USING.
DO NOT REMOVE THIS TAG.

NOTE: Microwave heating times vary – **DO NOT** overheat. **DO NOT** use convection function of your microwave oven.

INSTRUCTIONS FOR USE:

- **ALWAYS** heat on clean empty microwave safe dish. Lay as flat as possible and distribute contents of BED BUDDY evenly. If microwave **DOES NOT** have a turntable, rotate the pack 90 degrees halfway through the heating time. To ensure an even temperature, it is recommended to turn product over halfway through heating.
- Microwave BED BUDDY at standard power to reach a therapeutic temperature range of 40°C - 45°C (104°F - 113°F). Use the following chart for heating times based on microwave wattage:

600 W	900 W	1200 W
90 seconds	60 seconds	45 seconds

- **DO NOT** leave BED BUDDY unattended while heating.
- Following heating, allow product to sit in microwave for 1 minute before use. **ALWAYS** touch-test by shifting contents to check for hot spots before use. Product should feel **WARM** and **MOIST, NOT HOT**.
- As the product cools during use, the device may be reheated for 15 seconds to maintain a therapeutic temperature range. The product should **NOT** be reheated more than once every 10 minutes of use (15 seconds based on 900 W microwave).
- Contamination by food or liquids can cause overheating resulting in burns to the skin.
- Excessive heating and prolonged skin contact can cause burns.
- Continually shift the position of the BED BUDDY when in use.

FOR COLD THERAPY:

- Place in a sealed plastic bag and put in freezer for at least 8 hours.
- Apply for a maximum of 15-20 minutes. Wait one hour before reapplying, or as directed by a healthcare provider.

FOR MOIST HEAT THERAPY: Apply for a maximum of 15-20 minutes. Repeat every 2 hours or as directed by a healthcare provider. Allow product to return to room temperature before reheating.

PRODUCT CARE: **NEVER** submerge this BED BUDDY product in water. Spot clean by wiping with a damp cloth and let dry thoroughly. **DO NOT** lie on, or place heavy objects on top of pack. Discard product if torn and leaking.

CAUTION: **DO NOT** use while sleeping or under the influence of mind-altering drugs, alcohol or while suffering from dizziness. **DO NOT** use this product on persons with nerve damage, circulatory problems, impaired skin sensation, or with damaged or broken skin, or on persons unable to remove this product such as infants, children, and some older persons. Adult supervision is recommended when using on children. Consult physician before use if pregnant or any medical conditions exist. **NEVER** place a heated BED BUDDY over the eyes.

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