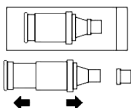
AIRIAL CHAMBER

COLLAPSIBLE HOLDING CHAMBER FOR METER DOSE INHALERS

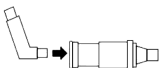
Model # MQ8001

1


Remove AIRIAL CHAMBER from package. Extend to full length into locked position and remove cap.

2


Remove MDI canister mouthpiece cover. Insert MDI canister mouthpiece into the elastic opening at the back of the chamber.

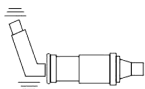3


Shake MDI canister and AIRIAL CHAMBER assembly 3-4 times.

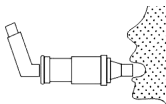4


Put Aerial Chamber mouthpiece between teeth. Close lips around mouthpiece.

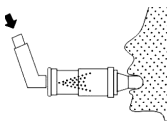5

Exhale normally. Press down on MDI canister to release medication.

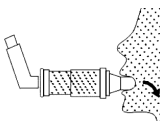6

Take a slow, deep breath through the mouthpiece. Hold breath for 5-10 seconds. Exhale through the mouthpiece. Take a second breath to ensure that all medication has been consumed.

Cleaning Instructions: Wash the Aerial Chamber in warm soapy water. Rinse with clean water and set aside to dry.

Precautions: Over time, the elastic rear opening and/or one-way valves may harden and not work effectively. Replace with a new Aerial Chamber.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

Made in China

Manufactured For: Drive Medical
99 Seaview Blvd.
Port Washington, NY 11050
Tel: (866) 694-5085

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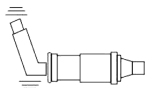
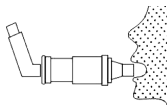
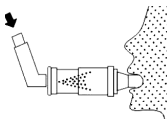
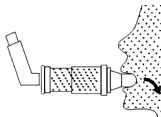
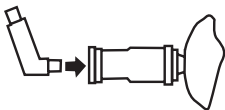
www.drivemedical.com

AIRIAL CHAMBER

HOLDING CHAMBER FOR METER DOSE INHALERS


Model # MQ8100

1


Remove the caps from the chambered and metered-dose inhaler. Shake the metered-dose inhaler vigorously. Insert the mouthpiece of the inhaler into the rubber-sealed end of the chamber.

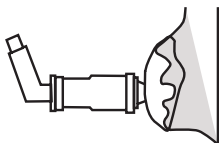2


Have your child stand or sit up straight. Put the mask on the child's face, covering both the nose and mouth. The mask must be pressed to your child's face to assure that the medicine can get to the child's lungs.

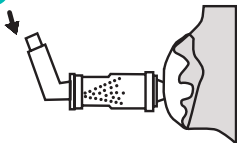3

Depress the metered-dose inhaler once. Have your child breathe in and out slowly through the valved holding chamber for at least 5 breaths.

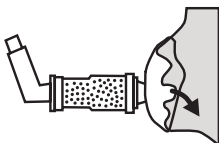4

When your healthcare practitioner prescribes additional puffs, wait 1 minute and repeat.

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