



WALSLIDE SLIM

- lightweight channel attaches to wall studs
- anchor (included) moves along center channel and locks in place by tightening with screwdriver
- adjust anchor to any height along 36" channel
- works with any elastic band or tubing

Instructions:

- Securely attach the top, center, and bottom of WalSlide Slim to studs in the wall. Make sure unit is securely attached to wall before using.
- To reposition WalSlide Slim sliding anchor along center groove loosen both screws above and below eye-bolt, position sliding anchor to desired height, retighten both screws.
- Band or tubing should be fastened to sliding anchor assembly with loop-stirrup anchor.

parts:

- 10-5070 - WalSlide Slim with sliding anchor, snap hook and loop stirrup
- 10-5071 - additional sliding anchor with snap hook
- 10-5310 - additional loop stirrup anchor





exercise band and tubing sold separately

mounting hardware sold separately

CAUTION:

- Consult your healthcare professional before beginning any exercise program
- Check that the WalSlide Slim is securely attached to the wall before each use
- Use the WalSlide slim with elastic exercise band and tubing only

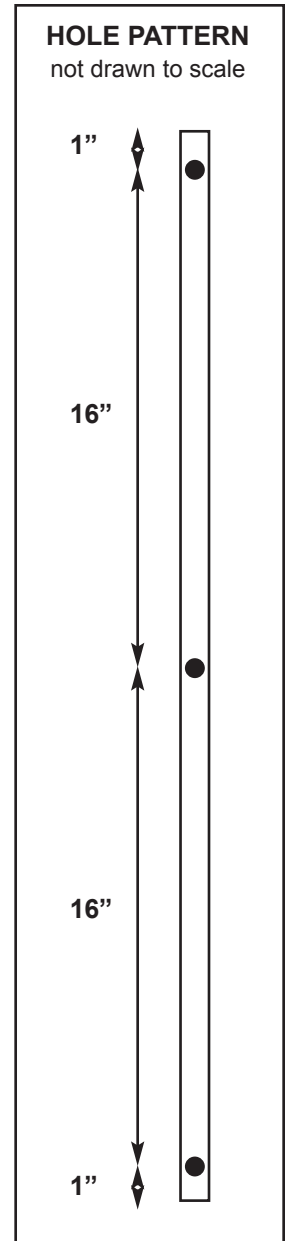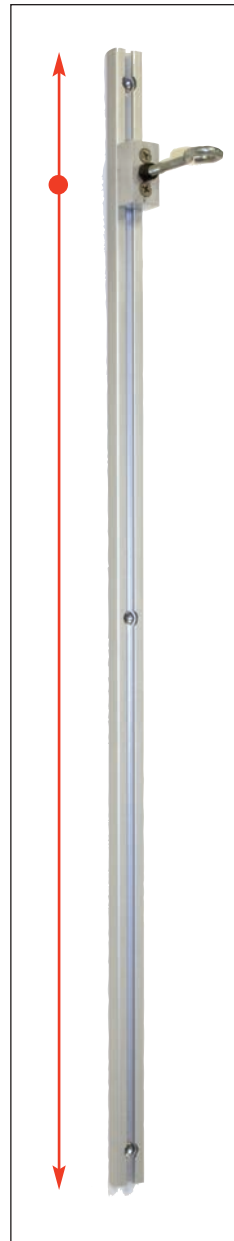

Fabrication Enterprises Inc.
P.O. Box 1500
White Plains, NY 10602 (USA)
tel: 1-914-345-9300 fax: 1-914-345-9800



Authorized CE representative:
RMS UK Ltd.
28 Trinity Road
Nailsea, Somerset BS48 4NU (UK)

10-5070



Cando, and WalSlide Slim are trademarks of Goldberg ©2011 FEI, all rights reserved

www.Fab-Ent.com