

OUR INNOVATIVE BREAST SHAPER PORTFOLIO

A portfolio of choices — more shapes, sizes, and thicknesses for a personalized fit.

	NE	EW	Former A thickness	Former B thickness	NEW	Carried over from former portfolio		
THICKNESS	TH	IIN	MEDIUM	VOLUME	SPECIAL			
CROSS-CUP DIFFERENCE	< 1 cm (or 3/8"	1-2 cm or 3/8" - 3/4"	> 2 cm or 3/4"				
SHAPE	OVAL DELTA		DELTA	DELTA	ELLIPSE	VARIA		
BALANCE CONTACT comfort+			Balance Contact MD 229 Sizes 2 - 12	Balance Contact VD 230 Sizes 3/4 - 9/10		Balance Contact SV 286 Sizes S - XL		
BALANCE NATURA				Balance Natura Light VD 221 Sizes 3/4 - 11/12				
comfort+	Balance Natura TO 227 Sizes S - XL	Balance Natura TD 217 Sizes 1 - 10	Balance Natura MD 220 Sizes 2 - 12	Balance Natura VD 222 Sizes 3/4 - 9/10	Balance Natura SE 231 Sizes S - XL	Balance Natura SV 285 Sizes S - XL		
BALANCE ESSENTIAL				Balance Essential Light VD 224 Sizes 3/4 - 11/12 Tawny 5/6 - 11/12				
	Balance Essential TO 228 Sizes S - XL	Balance Essential TD 218 Sizes 1 - 10	Balance Essential MD 223 Sizes 2 - 12 Tawny 4 - 10	Balance Essential VD 225 Sizes 3/4 - 9/10	Balance Essential SE 232 Sizes S - XL			

FINDING THE RIGHT SIZE SHAPER FOR HER


Use the charts below as a guide to finding a shaper thickness and size to start your fitting.

If your customer requires an OVAL or DELTA shaper, use the cross-cup difference method to find the thickness needed.

The cross-cup measurement is the width between the front center of a properly fitting bra (at her sternum) to the side seam.

- 1. With her bra on and her arms down, measure cross-cup on her non-surgery side.
- 2. Then, measure cross-cup on her surgery side.
- 3. Subtract the measurement of step 2 from the measurement of step 1. This final number is the Cross-Cup Difference (CCD).
- 4. Locate the measurement on the chart below to find which shaper thickness to try first.

TIP: Use the metric side of your tape measure and measure in centimeters (cm). This will give you the most accurate measurement.

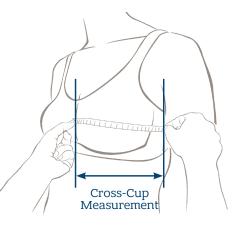

CROSS-CUP DIFFERENCE (CCD)	SHAPER SIZES AVAILABLE												
< 1 cm or 3/8"	OVAL			S		M		L		XL			
THIN	DELTA	1	2	3	4	5	6	7	8	9	10		
1 - 2 cm or 3/8" - 3/4" MEDIUM	DELTA		2	3	4	5	6	7	8	9	10	11	12
> 2 cm or 3/4" VOLUME DELTA				3/4		5/6		7/8		9/10		11/12	

Use the sizing chart below to find the right size of any style shaper to start your fitting.

Find the bra cup and band size, then follow the column down to the bottom row to find the shaper size.

CUP SIZE	BAND SIZE											
AA	32	34	36	38	40	42	44	46	48	50	52	
А	30	32	34	36	38	40	42	44	46	48	50	52
В		30	32	34	36	38	40	42	44	46	48	50
С			30	32	34	36	38	40	42	44	46	48
D				30	32	34	36	38	40	42	44	46
DD					30	32	34	36	38	40	42	44
DDD						30	32	34	36	38	40	42
G									34	36	38	40
	1	2	3	4	5	6	7	8	9	10	11	12
SHAPER SIZES			S		М		L		XL			
		3/4		/4	5/6		7/8		9/10		11/12	

FOUR SHAPES THAT OFFER LOTS OF OPTIONS

Surgery types, sites and the amount of missing tissue vary from woman to woman. With multiple shapes and sizes, you can find the ideal solution for her.









BALANCE FIT KITS

Ask your Account Manager about these tools designed to help support your fittings.



SINGLE FIT KIT NATURA MD 250

Sizes 2, 4, 6, 8, 10



SINGLE FIT KIT NATURA VD 251

Natura Sizes 5/6, 7/8 Natura Light Sizes 9/10, 11/12



PREMIUM FIT KIT 245

17 shapers in different sizes and a sizing chart



WE'RE HERE FOR YOU.